
# CompoMat G5 *Plus*

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Edição: 3/01.18

Art.-Nº.: M723351

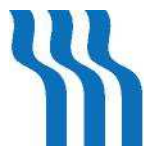**FRESENIUS  
KABI**

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Aviso

## 13.4 Certificados

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	
<b><u>EC DECLARATION OF CONFORMITY</u></b>	
Annex VII of Directive 93/42/EEC	
<b>CompoMat G5 Plus / CompoMat G5 / CompoMat G5 US RCC-Scale / PLS-Scale</b> (Product name)	
9026501 / 9025501 / 9025561 9026511 / 9026521 / 9025511 / 9025521 (Article number)	
xCQAxxxx / xCPTxxxx xRTAxxxx / xPTAxxxx / xRSTxxxx / xPSTxxxx (Serial number)	
Class <input checked="" type="checkbox"/> I	
We <b>Fresenius Kabi AG</b> <b>61346 Bad Homburg, Germany</b>	
<p>manufacturer of the above products, hereby declare under our sole responsibility that the referenced products comply with all relevant provisions of Directive 93/42/EEC, as amended by 2007/47/EC, and its transposition into national laws. The products comply with the essential requirements of Annex I, further applicable standards and/or other normative documents as listed in the applicable technical documentation. All supporting documentation is kept under the premises of the manufacturer.</p>	
	
<u>Friedberg, 06 March 2018</u> Place and date of issue	<u>Dieter Fries, Director Regulatory Affairs &amp; QA, BU TCT</u> Name (printed letters), position and signature of authorized person
Valid starting with the original date of the document until product change	

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MEDICAL ELECTRIC EQUIPMENT  
WITH RESPECT TO ELECTRICAL SHOCK,  
FIRE AND MECHANICAL HAZARDS ONLY  
IN ACCORDANCE WITH UL 60601-1,  
ANSI/AAMI ES60601-1 (2005),  
ANSI/AAMI ES60601-1 (2005) + AMD 1 (2012)  
and CAN/CSA C22.2 No. 60601-1  
(2008 and 2014)  
38PX

### 13.4.3 Software de código aberto

O aparelho usa software de código aberto

GNU General Public License

<http://www.gnu.org/licenses/gpl.html>

GNU Lesser General Public License

<http://www.gnu.org/copyleft/lesser.html>

# 14 License

## 14.1 License information

This product contains software (including firmware) originating from Fresenius Kabi AG and its suppliers and also contains software from the open source community, especially libraries licensed under GPL 2.0, LGPL 2 and LGPL 2.1. Any software originating from Fresenius Kabi AG and its suppliers is licensed under the Software License Agreement signed by you and provided for reference purposes in chapter 14.7.

Any software from the open source community is licensed under the specific license terms applicable to that software as specified below.

Where such specific license terms entitle you to the source code of such software and, in case of libraries licensed under LGPL 2 or LGPL 2.1, to the object code of the work that software is available from Technical Support Medical Device, Fresenius HemoCare GmbH, Grüner Weg 10, 61169 Friedberg, upon request only for the cost of transmitting the code to you via mail for at least three years from the purchase date of this product.

The following open source components are inter alia contained in this product:

Includes	License	Comment
arpa/inet.h	lgpl-2.1-plus	<p data-bbox="480 320 1358 389">/* Copyright (C) 1997, 1999, 2000, 2001, 2004 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="480 456 1262 584">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="480 651 1326 779">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="480 846 1257 965">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>
fcntl.h	lgpl-2.1-plus	<p data-bbox="480 976 1437 1046">/* Copyright (C) 1991,1992,1994-2001,2003,2004 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="480 1113 1262 1240">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="480 1308 1326 1435">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="480 1503 1257 1621">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
gdk/gdkx.h	lgpl-2.0-plus	<pre> /* GDK - The GIMP Drawing Kit * Copyright (C) 1995-1997 Peter Mattis, Spencer Kimball and Josh MacDonald * * This library is free software; you can redistribute it and/or * modify it under the terms of the GNU Lesser General Public * License as published by the Free Software Foundation; either * version 2 of the License, or (at your option) any later version. * * This library is distributed in the hope that it will be useful, * but WITHOUT ANY WARRANTY; without even the implied warranty of * MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. * See the GNU Lesser General Public License for more details. * * You should have received a copy of the GNU Lesser General Public * License along with this library; if not, write to the * Free Software Foundation, Inc., 59 Temple Place - Suite 330, * Boston, MA 02111-1307, USA. */ /* * Modified by the GTK+ Team and others 1997-2000. See the AUTHORS * file for a list of people on the GTK+ Team. See the ChangeLog * files for a list of changes. These files are distributed with * GTK+ at <a href="http://ftp.gtk.org/pub/gtk/">http://ftp.gtk.org/pub/gtk/</a>. */ </pre>
glib/gprintf.h	lgpl-2.0-plus	<pre> /* GLIB - Library of useful routines for C programming * Copyright (C) 1995-1997, 2002 Peter Mattis, Red Hat, Inc. * * This library is free software; you can redistribute it and/or * modify it under the terms of the GNU Lesser General Public * License as published by the Free Software Foundation; either * version 2 of the License, or (at your option) any later version. * * This library is distributed in the hope that it will be useful, * but WITHOUT ANY WARRANTY; without even the implied warranty of * MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. * See the GNU Lesser General Public License for more details. * * You should have received a copy of the GNU Lesser General Public * License along with this library; if not, write to the * Free Software Foundation, Inc., 59 Temple Place - Suite 330, * Boston, MA 02111-1307, USA. */ </pre>

Includes	License	Comment
gtk/gtk.h	lgpl-2.0-plus	<pre> /* GTK - The GIMP Toolkit  * Copyright (C) 1995-1997 Peter Mattis, Spencer Kimball and Josh MacDonald  *  * This library is free software; you can redistribute it and/or  * modify it under the terms of the GNU Lesser General Public  * License as published by the Free Software Foundation; either  * version 2 of the License, or (at your option) any later version.  *  * This library is distributed in the hope that it will be useful,  * but WITHOUT ANY WARRANTY; without even the implied warranty of  * MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE.  * See the GNU Lesser General Public License for more details.  *  * You should have received a copy of the GNU Lesser General Public  * License along with this library; if not, write to the  * Free Software Foundation, Inc., 59 Temple Place - Suite 330,  * Boston, MA 02111-1307, USA.  */ </pre>
math.h	lgpl-2.1-plus	<pre> /* Declarations for math functions.  * Copyright (C) 1991-1993,1995-1999,2001,2002,2004 Free Software  * Foundation, Inc.  *  * This file is part of the GNU C Library.  *  * The GNU C Library is free software; you can redistribute it and/or  * modify it under the terms of the GNU Lesser General Public  * License as published by the Free Software Foundation; either  * version 2.1 of the License, or (at your option) any later version.  *  * The GNU C Library is distributed in the hope that it will be useful, but  * WITHOUT ANY WARRANTY; without even the implied warranty of  * MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE.  * See the GNU Lesser General Public License for more details.  *  * You should have received a copy of the GNU Lesser General  * Public License along with the GNU C Library; if not, write to the  * Free Software Foundation, Inc., 59 Temple Place, Suite 330,  * Boston, MA 02111-1307 USA. */ </pre>

Includes	License	Comment
net/if.h	lgpl-2.1-plus	<p data-bbox="531 264 1497 383">/* net/if.h -- declarations for inquiring about network interfaces Copyright (C) 1997,98,99,2000,2001 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="531 456 1497 575">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="531 649 1497 768">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="531 842 1497 965">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>
netinet/in.h	lgpl-2.1-plus	<p data-bbox="531 969 1497 1043">/* Copyright (C) 1991-2001, 2003, 2004 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="531 1117 1497 1236">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="531 1310 1497 1429">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="531 1503 1497 1619">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
stdarg.h	gpl-2.0-gcc-compiler-exception	<p data-bbox="480 266 1406 293">/* Copyright (C) 1989, 1997, 1998, 1999, 2000 Free Software Foundation, Inc.</p> <p data-bbox="480 360 756 387">This file is part of GCC.</p> <p data-bbox="480 454 1267 577">GCC is free software; you can redistribute it and/or modify it under the terms of the GNU General Public License as published by the Free Software Foundation; either version 2, or (at your option) any later version.</p> <p data-bbox="480 645 1398 768">GCC is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU General Public License for more details.</p> <p data-bbox="480 835 1302 965">You should have received a copy of the GNU General Public License along with GCC; see the file COPYING. If not, write to the Free Software Foundation, 59 Temple Place - Suite 330, Boston, MA 02111-1307, USA. */</p> <p data-bbox="480 1032 1334 1211">/* As a special exception, if you include this header file into source files compiled by GCC, this header file does not by itself cause the resulting executable to be covered by the GNU General Public License. This exception does not however invalidate any other reasons why the executable file might be covered by the GNU General Public License. */</p>
stdio.h	lgpl-2.1-plus	<p data-bbox="480 1225 1027 1252">/* Define ISO C stdio on top of C++ iostreams.</p> <p data-bbox="480 1274 1366 1301">Copyright (C) 1991,1994-2002,2003,2004 Free Software Foundation, Inc.</p> <p data-bbox="480 1323 930 1350">This file is part of the GNU C Library.</p> <p data-bbox="480 1417 1267 1541">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="480 1608 1318 1731">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="480 1798 1302 1921">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
stdlib.h	lgpl-2.1-plus	<p data-bbox="531 271 1326 342">/* Copyright (C) 1991-2002, 2003, 2004 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="557 409 1318 533">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="557 600 1378 723">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="557 790 1310 913">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>
string.h	lgpl-2.1-plus	<p data-bbox="531 925 1396 996">/* Copyright (C) 1991-1993, 1995-2003, 2004 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="557 1064 1318 1187">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="557 1254 1378 1377">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="557 1444 1310 1568">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
sys/ioctl.h	lgpl-2.1-plus	<p data-bbox="485 271 1326 342">/* Copyright (C) 1991, 92, 93, 94, 96, 98, 99 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="507 409 1262 533">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="507 600 1326 723">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="507 790 1257 913">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>
sys/mount.h	lgpl-2.1-plus	<p data-bbox="485 925 1129 949">/* Header file for mounting/unmount Linux filesystems.</p> <p data-bbox="485 969 1430 1041">Copyright (C) 1996,1997,1998,1999,2000,2004 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="507 1108 1262 1232">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="507 1299 1326 1422">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="507 1489 1257 1612">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
sys/socket.h	lgpl-2.1-plus	<p data-bbox="529 264 1497 297">/* Declarations of socket constants, types, and functions.</p> <p data-bbox="529 309 1497 342">Copyright (C) 1991,92,1994-2001,2003 Free Software Foundation, Inc.</p> <p data-bbox="529 353 1497 387">This file is part of the GNU C Library.</p> <p data-bbox="529 454 1497 577">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="529 645 1497 768">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="529 835 1497 958">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>
sys/stat.h	lgpl-2.1-plus	<p data-bbox="529 969 1497 1003">/* Copyright (C) 1991,1992,1995-2002,2003,2004 Free Software Foundation, Inc.</p> <p data-bbox="529 1014 1497 1048">This file is part of the GNU C Library.</p> <p data-bbox="529 1115 1497 1238">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="529 1305 1497 1429">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="529 1496 1497 1619">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
sys/types.h	lgpl-2.1-plus	<p data-bbox="481 264 1406 297">/* Copyright (C) 1991,1992,1994,1995,1996,1997,1998,1999,2000,2001,2002</p> <p data-bbox="547 313 916 342">Free Software Foundation, Inc.</p> <p data-bbox="507 358 946 387">This file is part of the GNU C Library.</p> <p data-bbox="507 456 1262 580">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="507 647 1326 770">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="507 837 1257 960">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>
sys/un.h	lgpl-2.1-plus	<p data-bbox="481 976 1278 1043">/* Copyright (C) 1991, 1995, 1996, 2001 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="507 1113 1262 1236">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="507 1303 1326 1426">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="507 1494 1257 1617">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
time.h	lgpl-2.1-plus	<p data-bbox="531 271 1503 342">/* Copyright (C) 1991-1999,2000,2001,2002,2003 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="531 409 1503 533">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="531 600 1503 723">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="531 790 1503 913">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>
unistd.h	lgpl-2.1-plus	<p data-bbox="531 925 1503 996">/* Copyright (C) 1991-2002, 2003, 2004 Free Software Foundation, Inc. This file is part of the GNU C Library.</p> <p data-bbox="531 1064 1503 1187">The GNU C Library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.</p> <p data-bbox="531 1254 1503 1377">The GNU C Library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.</p> <p data-bbox="531 1444 1503 1570">You should have received a copy of the GNU Lesser General Public License along with the GNU C Library; if not, write to the Free Software Foundation, Inc., 59 Temple Place, Suite 330, Boston, MA 02111-1307 USA. */</p>

Includes	License	Comment
X11/Xlib.h	x11-opengroup	<pre> /* \$XdotOrg: xc/lib/X11/Xlib.h,v 1.2 2004/04/23 18:43:24 eich Exp \$ */ /* \$Xorg: Xlib.h,v 1.6 2001/02/09 02:03:38 xorgcvs Exp \$ */ /*  Copyright 1985, 1986, 1987, 1991, 1998 The Open Group  Permission to use, copy, modify, distribute, and sell this software and its documentation for any purpose is hereby granted without fee, provided that the above copyright notice appear in all copies and that both that copyright notice and this permission notice appear in supporting documentation.  The above copyright notice and this permission notice shall be included in all copies or substantial portions of the Software.  THE SOFTWARE IS PROVIDED "AS IS", WITHOUT WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO THE WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE AND NO-NINFRINGEMENT. IN NO EVENT SHALL THE OPEN GROUP BE LIABLE FOR ANY CLAIM, DAMAGES OR OTHER LIABILITY, WHETHER IN AN ACTION OF CONTRACT, TORT OR OTHERWISE, ARISING FROM, OUT OF OR IN CONNECTION WITH THE SOFTWARE OR THE USE OR OTHER DEALINGS IN THE SOFTWARE.  Except as contained in this notice, the name of The Open Group shall not be used in advertising or otherwise to promote the sale, use or other dealings in this Software without prior written authorization from The Open Group.  */ /* \$XFree86: xc/lib/X11/Xlib.h,v 3.25 2003/11/17 22:20:10 dawes Exp \$ */ /* * Xlib.h - Header definition and support file for the C subroutine * interface library (Xlib) to the X Window System Protocol (V11). * Structures and symbols starting with "_" are private to the library. */ </pre>

## 14.2 Packages

Package name	License
elinos-rpm	GPL
elinos-apache-arm_v4le-glibc-2.3.4	Apache
elinos-ash-arm_v4le-glibc-2.3.4	BSD
elinos-atk-arm_v4le-glibc-2.3.4	GPLv2
elinos-bash-arm_v4le-glibc-2.3.4	GPLv2
elinos-bc-arm_v4le-glibc-2.3.4	GPLv2
elinos-berkeleydb-arm_v4le-glibc-2.3.4	Sleepycat Software
elinos-bind-arm_v4le-glibc-2.3.4	Internet Software Consortium
elinos-binutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-boot-blob	SYSGO AG
elinos-boot-zImage	SYSGO AG
elinos-bsp-Tuxscreen	GPL
elinos-bsp-at91	GPL
elinos-bsp-cdb89712	GPL
elinos-bsp-impA7	GPL
elinos-bsp-lart	GPL
elinos-bsp-mp-1x	GPL
elinos-bsp-pcm022	GPL
elinos-bsp-pcm027	GPL
elinos-bsp-pcm7210	GPL
elinos-bsp-qemu-integrator	GPL
elinos-busybox-arm_v4le-glibc-2.3.4	GPLv2
elinos-bzip2-arm_v4le-glibc-2.3.4	Julian R Seward
elinos-c++-arm_v4le-glibc-2.3.4	GPLv2
elinos-can-festival-arm_v4le-glibc-2.3.4	LGPL
elinos-clflasher-arm_v4le-glibc-2.3.4	GPL
elinos-cognito	SYSGO AG
elinos-coreutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-cron-arm_v4le-glibc-2.3.4	BSD
elinos-demo-applications	SYSGO AG
elinos-demo-busybox	SYSGO AG
elinos-demo-codeo	SYSGO AG
elinos-demo-cognito	SYSGO AG
elinos-demo-empty	SYSGO AG
elinos-demo-gdbserver-net	SYSGO AG
elinos-demo-gdbserver-serial	SYSGO AG
elinos-demo-gtk	SYSGO AG
elinos-demo-hello	SYSGO AG
elinos-demo-inet-services	SYSGO AG
elinos-demo-jffs	SYSGO AG
elinos-demo-ltt	SYSGO AG
elinos-demo-networking	SYSGO AG
elinos-demo-nfsmount	SYSGO AG
elinos-demo-nfsroot	SYSGO AG
elinos-demo-qte	SYSGO AG
elinos-demo-tempo	SYSGO AG

Package name	License
elinos-demo-ultra-embedded	SYSGO AG
elinos-demo-webserver	SYSGO AG
elinos-demo-x	SYSGO AG
elinos-devfsd-arm_v4le-glibc-2.3.4	GPLv2
elinos-dhcp-arm_v4le-glibc-2.3.4	GPLv2
elinos-diffutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-dosfstools-arm_v4le-glibc-2.3.4	GPLv2
elinos-e2fsprogs-arm_v4le-glibc-2.3.4	GPLv2
elinos-eclipse-codeo	SYSGO AG
elinos-eclipse-core	SYSGO AG
elinos-eclipse-doc	SYSGO AG
elinos-eclipse-pbuilder	SYSGO AG
elinos-eclipse-remote	SYSGO AG
elinos-eclipse-systemviewer	SYSGO AG
elinos-efence-arm_v4le-glibc-2.3.4	GPLv2
elinos-elk	SYSGO AG
elinos-expat-arm_v4le-glibc-2.3.4	Thai Open Source Software Center Ltd and Clark Cooper
elinos-fbset-arm_v4le-glibc-2.3.4	GPLv2
elinos-feature-config	SYSGO AG
elinos-feature-dprobes	SYSGO AG
elinos-features	SYSGO AG
elinos-findutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-flex-arm_v4le-glibc-2.3.4	BSD
elinos-fontconfig-arm_v4le-glibc-2.3.4	MIT, Keith Packard
elinos-freetype-arm_v4le-glibc-2.3.4	GPLv2, Free Type License
elinos-g77-arm_v4le-glibc-2.3.4	GPLv2
elinos-gawk-arm_v4le-glibc-2.3.4	GPLv2
elinos-gcc-arm_v4le-glibc-2.3.4	GPLv2
elinos-gdb-client-arm_v4le-glibc-2.3.4	GPLv2
elinos-glib-arm_v4le-glibc-2.3.4	GPLv2
elinos-glibc-arm_v4le-glibc-2.3.4	LGPL
elinos-gmp-arm_v4le-glibc-2.3.4	GPLv2
elinos-grep-arm_v4le-glibc-2.3.4	GPLv2
elinos-gtk-arm_v4le-glibc-2.3.4	GPLv2
elinos-gtkfonts-arm_v4le-glibc-2.3.4	GPLv2
elinos-gzip-arm_v4le-glibc-2.3.4	GPLv2
elinos-hdparm-arm_v4le-glibc-2.3.4	BSD, Mark Lord
elinos-hotplug-arm_v4le-glibc-2.3.4	GPLv2
elinos-images-u-boot-arm_v4le-glibc-2.3.4	GPLv2
elinos-inetutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-iproute2-arm_v4le-glibc-2.3.4	GPLv2
elinos-ipsec-tools-arm_v4le-glibc-2.3.4	GPLv2
elinos-iptables-arm_v4le-glibc-2.3.4	GPLv2
elinos-iputils-arm_v4le-glibc-2.3.4	BSD
elinos-jpeg-arm_v4le-glibc-2.3.4	GPLv2
elinos-kaffe-arm_v4le-glibc-2.3.4	LGPLv2.1
elinos-kaffe-host	LGPLv2.1
elinos-kbd-arm_v4le-glibc-2.3.4	GPLv2
elinos-kbd-host	GPLv2

Package name	License
elinos-ksymoops-arm_v4le-glibc-2.3.4	GPLv2
elinos-lart-tools-arm_v4le-glibc-2.3.4	GPL
elinos-less-arm_v4le-glibc-2.3.4	GPLv2
elinos-libgdbm-arm_v4le-glibc-2.3.4	GPLv2
elinos-libpcap-arm_v4le-glibc-2.3.4	BSD
elinos-libpng-arm_v4le-glibc-2.3.4	libpng License
elinos-libtermcap-arm_v4le-glibc-2.3.4	LGPLv2.1
elinos-lighttpd-arm_v4le-glibc-2.3.4	BSD
elinos-linux-arm-2.4.31	GPLv2
elinos-linux-arm-2.6.15	GPLv2
elinos-linuxheaders-arm_v4le-glibc-2.3.4	GPLv2
elinos-ltrace-arm_v4le-glibc-2.3.4	GPLv2
elinos-lxdialog	GPLv2
elinos-lzo-arm_v4le-glibc-2.3.4	GPLv2
elinos-madplay-arm_v4le-glibc-2.3.4	GPLv2
elinos-make	GPLv2
elinos-mdadm-arm_v4le-glibc-2.3.4	GPLv2
elinos-mkafs	SYSGO AG
elinos-module-init-tools-arm_v4le-glibc-2.3.4	GPLv2
elinos-modutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-modutils-host	GPLv2
elinos-mounta-arm_v4le-glibc-2.3.4	SYSGO AG - open source
elinos-mtd-utils-arm_v4le-glibc-2.3.4	GPLv2
elinos-nano-arm_v4le-glibc-2.3.4	GPLv2
elinos-nat-binutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-nat-gdb-arm_v4le-glibc-2.3.4	GPLv2
elinos-nat-gdb-server-arm_v4le-glibc-2.3.4	GPLv2
elinos-nat-make-arm_v4le-glibc-2.3.4	GPLv2
elinos-ncurses-arm_v4le-glibc-2.3.4	MIT
elinos-net-snmp-arm_v4le-glibc-2.3.4	BSD
elinos-net-tools-arm_v4le-glibc-2.3.4	GPLv2
elinos-netcat	None
elinos-netcat-arm_v4le-glibc-2.3.4	GPLv2
elinos-netconfig-arm_v4le-glibc-2.3.4	SYSGO AG - open source
elinos-nfs-utils-arm_v4le-glibc-2.3.4	GPLv2
elinos-ntp-arm_v4le-glibc-2.3.4	GPLv2
elinos-openssh-arm_v4le-glibc-2.3.4	GPLv2
elinos-openssl-arm_v4le-glibc-2.3.4	GPLv2
elinos-openvpn-arm_v4le-glibc-2.3.4	GPLv2
elinos-p2linux-arm_v4le-glibc-2.3.4	GPLv2
elinos-p2linux-example	GPLv2
elinos-pango-arm_v4le-glibc-2.3.4	GPLv2
elinos-pciutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-pcre-arm_v4le-glibc-2.3.4	BSD
elinos-peak-linux-driver-arm_v4le-glibc-2.3.4	GPLv2
elinos-peak-linux-driver-common	GPLv2
elinos-pkgconfig-arm_v4le-glibc-2.3.4	GPLv2
elinos-portmap-arm_v4le-glibc-2.3.4	GPLv2
elinos-pppd-arm_v4le-glibc-2.3.4	GPLv2
elinos-procinfo-arm_v4le-glibc-2.3.4	GPLv2

Package name	License
elinos-procps-arm_v4le-glibc-2.3.4	LGPL, GLP, BSD-like
elinos-productkeyjni	SYSGO AG
elinos-psmisc-arm_v4le-glibc-2.3.4	GPLv2
elinos-python-arm_v4le-glibc-2.3.4	GPLv2
elinos-qt-embedded3-arm_v4le-glibc-2.3.4	GPLv2/QPL
elinos-rpmconfig	SYSGO AG
elinos-rxtx-arm_v4le-glibc-2.3.4	GPLv2 (programs), relaxed LGPLv2.1 (libraries), and public domain (docs)
elinos-schedutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-scripts	SYSGO AG
elinos-sed-arm_v4le-glibc-2.3.4	GPLv2
elinos-setserial-arm_v4le-glibc-2.3.4	GPLv2
elinos-sqlite-arm_v4le-glibc-2.3.4	GPLv2
elinos-sslwrap-arm_v4le-glibc-2.3.4	GPLv2
elinos-strace-arm_v4le-glibc-2.3.4	GPLv2
elinos-strongswan-arm_v4le-glibc-2.3.4	GPLv2
elinos-sudo-arm_v4le-glibc-2.3.4	BSD-style, Todd C. Miller
elinos-sysfsutils-arm_v4le-glibc-2.3.4	GPLv2
elinos-sysklogd-arm_v4le-glibc-2.3.4	GPLv2
elinos-systembrowser-server-arm_v4le-glibc-2.3.4	SYSGO AG
elinos-sysvinit-arm_v4le-glibc-2.3.4	GPLv2, Other License(s), see package
elinos-tar-arm_v4le-glibc-2.3.4	GPLv2
elinos-tcp2term-arm_v4le-glibc-2.3.4	SYSGO AG
elinos-tcp_wrappers-arm_v4le-glibc-2.3.4	Other License(s), see package
elinos-tcpdump-arm_v4le-glibc-2.3.4	BSD
elinos-thttpd-arm_v4le-glibc-2.3.4	GPLv2
elinos-tiff-arm_v4le-glibc-2.3.4	BSD
elinos-time-arm_v4le-glibc-2.3.4	GPLv2
elinos-tinylogin-arm_v4le-glibc-2.3.4	GPLv2
elinos-tracedef-linux	SYSGO AG
elinos-traceroute-arm_v4le-glibc-2.3.4	BSD
elinos-tracetoolkit-arm_v4le-glibc-2.3.4	GPLv2
elinos-tracevisualizer	GPLv2
elinos-u-boot	GPLv2
elinos-udev-arm_v4le-glibc-2.3.4	GPLv2
elinos-unionfs-utils-arm_v4le-glibc-2.3.4	GPLv2
elinos-util-linux-arm_v4le-glibc-2.3.4	GPLv2, BSD
elinos-v2linux-arm_v4le-glibc-2.3.4	LGPLv2.1
elinos-v2linux-example	LGPLv2.1
elinos-validate	SYSGO AG
elinos-version	SYSGO AG
elinos-vim-arm_v4le-glibc-2.3.4	Other License(s), see package
elinos-wm2-arm_v4le-glibc-2.3.4	GPLv2
elinos-xorg-base-arm_v4le-glibc-2.3.4	GPL
elinos-xvt-arm_v4le-glibc-2.3.4	GPLv2
elinos-zlib-arm_v4le-glibc-2.3.4	GPLv2

## 14.3 Application libraries

libgtk-x11-2.0.so.0  
libgdk-x11-2.0.so.0  
libatk-1.0.so.0  
libgdk\_pixbuf-2.0.so.0  
[libtiff.so.3](#)  
[libjpeg.so.62](#)  
[libpng12.so.0](#)  
[libz.so.1](#)  
[libm.so.6](#)  
libpangoxft-1.0.so.0  
libpangox-1.0.so.0  
libpango-1.0.so.0  
libgobject-2.0.so.0  
libgmodule-2.0.so.0  
[libdl.so.2](#)  
libgthread-2.0.so.0  
libglib-2.0.so.0  
[libpthread.so.0](#)  
[libc.so.6](#)  
[libX11.so.6](#)

## 14.4 GNU GENERAL PUBLIC LICENSE Version 2, June 1991

Copyright (C) 1989, 1991 Free Software Foundation, Inc. 51 Franklin Street,  
Fifth Floor, Boston, MA 02110-1301, USA

Everyone is permitted to copy and distribute verbatim copies of this license document, but changing it is not allowed.

### Preamble

The licenses for most software are designed to take away your freedom to share and change it. By contrast, the GNU General Public License is intended to guarantee your freedom to share and change free software--to make sure the software is free for all its users. This General Public License applies to most of the Free Software Foundation's software and to any other program whose authors commit to using it. (Some other Free Software Foundation software is covered by the GNU Lesser General Public License instead.) You can apply it to your programs, too.

When we speak of free software, we are referring to freedom, not price. Our General Public Licenses are designed to make sure that you have the freedom to distribute copies of free software (and charge for this service if you wish), that you receive source code or can get it if you want it, that you can change the software or use pieces of it in new free programs; and that you know you can do these things.

To protect your rights, we need to make restrictions that forbid anyone to deny you these rights or to ask you to surrender the rights. These restrictions translate to certain responsibilities for you if you distribute copies of the software, or if you modify it.

For example, if you distribute copies of such a program, whether gratis or for a fee, you must give the recipients all the rights that you have. You must make sure that they, too, receive or can get the source code. And you must show them these terms so they know their rights.

We protect your rights with two steps: (1) copyright the software, and (2) offer you this license which gives you legal permission to copy, distribute and/or modify the software.

Also, for each author's protection and ours, we want to make certain that everyone understands that there is no warranty for this free software. If the software is modified by someone else and passed on, we want its recipients to know that what they have is not the original, so that any problems introduced by others will not reflect on the original authors' reputations.

Finally, any free program is threatened constantly by software patents. We wish to avoid the danger that redistributors of a free program will individually obtain patent licenses, in effect making the program proprietary. To prevent this, we have made it clear that any patent must be licensed for everyone's free use or not licensed at all.

The precise terms and conditions for copying, distribution and modification follow.

## **TERMS AND CONDITIONS FOR COPYING, DISTRIBUTION AND MODIFICATION**

**0.** This License applies to any program or other work which contains a notice placed by the copyright holder saying it may be distributed under the terms of this General Public License. The "Program", below, refers to any such program or work, and a "work based on the Program" means either the Program or any derivative work under copyright law: that is to say, a work containing the Program or a portion of it, either verbatim or with modifications and/or translated into another language. (Hereinafter, translation is included without limitation in the term "modification".) Each licensee is addressed as "you".

Activities other than copying, distribution and modification are not covered by this License; they are outside its scope. The act of running the Program is not restricted, and the output from the Program is covered only if its contents constitute a work based on the Program (independent of having been made by running the Program). Whether that is true depends on what the Program does.

1. You may copy and distribute verbatim copies of the Program's source code as you receive it, in any medium, provided that you conspicuously and appropriately publish on each copy an appropriate copyright notice and disclaimer of warranty; keep intact all the notices that refer to this License and to the absence of any warranty; and give any other recipients of the Program a copy of this License along with the Program.

You may charge a fee for the physical act of transferring a copy, and you may at your option offer warranty protection in exchange for a fee.

2. You may modify your copy or copies of the Program or any portion of it, thus forming a work based on the Program, and copy and distribute such modifications or work under the terms of Section 1 above, provided that you also meet all of these conditions:

- a) You must cause the modified files to carry prominent notices stating that you changed the files and the date of any change.
- b) You must cause any work that you distribute or publish, that in whole or in part contains or is derived from the Program or any part thereof, to be licensed as a whole at no charge to all third parties under the terms of this License.

**c)** If the modified program normally reads commands interactively when run, you must cause it, when started running for such interactive use in the most ordinary way, to print or display an announcement including an appropriate copyright notice and a notice that there is no warranty (or else, saying that you provide a warranty) and that users may redistribute the program under these conditions, and telling the user how to view a copy of this License.

(Exception: if the Program itself is interactive but does not normally print such an announcement, your work based on the Program is not required to print an announcement.)

These requirements apply to the modified work as a whole. If identifiable sections of that work are not derived from the Program, and can be reasonably considered independent and separate works in themselves, then this License, and its terms, do not apply to those sections when you distribute them as separate works. But when you distribute the same sections as part of a whole which is a work based on the Program, the distribution of the whole must be on the terms of this License, whose permissions for other licensees extend to the entire whole, and thus to each and every part regardless of who wrote it.

Thus, it is not the intent of this section to claim rights or contest your rights to work written entirely by you; rather, the intent is to exercise the right to control the distribution of derivative or collective works based on the Program.

In addition, mere aggregation of another work not based on the Program with the Program (or with a work based on the Program) on a volume of a storage or distribution medium does not bring the other work under the scope of this License.

**3.** You may copy and distribute the Program (or a work based on it, under Section 2) in object code or executable form under the terms of Sections 1 and 2 above provided that you also do one of the following:

- a) Accompany it with the complete corresponding machine-readable source code, which must be distributed under the terms of Sections 1 and 2 above on a medium customarily used for software interchange; or,
- b) Accompany it with a written offer, valid for at least three years, to give any third party, for a charge no more than your cost of physically performing source distribution, a complete machine-readable copy of the corresponding source code, to be distributed under the terms of Sections 1 and 2 above on a medium customarily used for software interchange; or,
- c) Accompany it with the information you received as to the offer to distribute corresponding source code. (This alternative is allowed only for noncommercial distribution and only if you received the program in object code or executable form with such an offer, in accord with Subsection b above.)

The source code for a work means the preferred form of the work for making modifications to it. For an executable work, complete source code means all the source code for all modules it contains, plus any associated interface definition files, plus the scripts used to control compilation and installation of the executable. However, as a special exception, the source code distributed need not include anything that is normally distributed (in either source or binary form) with the major components (compiler, kernel, and so on) of the operating system on which the executable runs, unless that component itself accompanies the executable.

If distribution of executable or object code is made by offering access to copy from a designated place, then offering equivalent access to copy the source code from the same place counts as distribution of the source code, even though third parties are not compelled to copy the source along with the object code.

**4.** You may not copy, modify, sublicense, or distribute the Program except as expressly provided under this License. Any attempt otherwise to copy, modify, sublicense or distribute the Program is void, and will automatically terminate your rights under this License.

However, parties who have received copies, or rights, from you under this License will not have their licenses terminated so long as such parties remain in full compliance.

5. You are not required to accept this License, since you have not signed it. However, nothing else grants you permission to modify or distribute the Program or its derivative works. These actions are prohibited by law if you do not accept this License. Therefore, by modifying or distributing the Program (or any work based on the Program), you indicate your acceptance of this License to do so, and all its terms and conditions for copying, distributing or modifying the Program or works based on it.

6. Each time you redistribute the Program (or any work based on the Program), the recipient automatically receives a license from the original licensor to copy, distribute or modify the Program subject to these terms and conditions. You may not impose any further restrictions on the recipients' exercise of the rights granted herein. You are not responsible for enforcing compliance by third parties to this License.

7. If, as a consequence of a court judgment or allegation of patent infringement or for any other reason (not limited to patent issues), conditions are imposed on you (whether by court order, agreement or otherwise) that contradict the conditions of this License, they do not excuse you from the conditions of this License. If you cannot distribute so as to satisfy simultaneously your obligations under this License and any other pertinent obligations, then as a consequence you may not distribute the Program at all. For example, if a patent license would not permit royalty-free redistribution of the Program by all those who receive copies directly or indirectly through you, then the only way you could satisfy both it and this License would be to refrain entirely from distribution of the Program.

If any portion of this section is held invalid or unenforceable under any particular circumstance, the balance of the section is intended to apply and the section as a whole is intended to apply in other circumstances.

It is not the purpose of this section to induce you to infringe any patents or other property right claims or to contest validity of any such claims; this section has the sole purpose of protecting the integrity of the free software distribution system, which is implemented by public license practices. Many people have made generous contributions to the wide range of software distributed through that system in reliance on consistent application of that system; it is up to the author/donor to decide if he or she is willing to distribute software through any other system and a licensee cannot impose that choice.

This section is intended to make thoroughly clear what is believed to be a consequence of the rest of this License.

8. If the distribution and/or use of the Program is restricted in certain countries either by patents or by copyrighted interfaces, the original copyright holder who places the Program under this License may add an explicit geographical distribution limitation excluding those countries, so that distribution is permitted only in or among countries not thus excluded. In such case, this License incorporates the limitation as if written in the body of this License.

9. The Free Software Foundation may publish revised and/or new versions of the General Public License from time to time. Such new versions will be similar in spirit to the present version, but may differ in detail to address new problems or concerns.

Each version is given a distinguishing version number. If the Program specifies a version number of this License which applies to it and "any later version", you have the option of following the terms and conditions either of that version or of any later version published by the Free Software Foundation. If the Program does not specify a version number of this License, you may choose any version ever published by the Free Software Foundation.

10. If you wish to incorporate parts of the Program into other free programs whose distribution conditions are different, write to the author to ask for permission. For software which is copyrighted by the Free Software Foundation, write to the Free Software Foundation; we sometimes make exceptions for this. Our decision will be guided by the two goals of preserving the free status of all derivatives of our free software and of promoting the sharing and reuse of software generally.

## NO WARRANTY

11. BECAUSE THE PROGRAM IS LICENSED FREE OF CHARGE, THERE IS NO WARRANTY FOR THE PROGRAM, TO THE EXTENT PERMITTED BY APPLICABLE LAW. EXCEPT WHEN OTHERWISE STATED IN WRITING THE COPYRIGHT HOLDERS AND/OR OTHER PARTIES PROVIDE THE PROGRAM "AS IS" WITHOUT WARRANTY OF ANY KIND, EITHER EXPRESSED OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE PROGRAM IS WITH YOU. SHOULD THE PROGRAM PROVE DEFECTIVE, YOU ASSUME THE COST OF ALL NECESSARY SERVICING, REPAIR OR CORRECTION.

12. IN NO EVENT UNLESS REQUIRED BY APPLICABLE LAW OR AGREED TO IN WRITING WILL ANY COPYRIGHT HOLDER, OR ANY OTHER PARTY WHO MAY MODIFY AND/OR REDISTRIBUTE THE PROGRAM AS PERMITTED ABOVE, BE LIABLE TO YOU FOR DAMAGES, INCLUDING ANY GENERAL, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING OUT OF THE USE OR INABILITY TO USE THE PROGRAM (INCLUDING BUT NOT LIMITED TO LOSS OF DATA OR DATA BEING RENDERED INACCURATE OR LOSSES SUSTAINED BY YOU OR THIRD PARTIES OR A FAILURE OF THE PROGRAM TO OPERATE WITH ANY OTHER PROGRAMS), EVEN IF SUCH HOLDER OR OTHER PARTY HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES.

END OF TERMS AND CONDITIONS

### How to Apply These Terms to Your New Programs

If you develop a new program, and you want it to be of the greatest possible use to the public, the best way to achieve this is to make it free software which everyone can redistribute and change under these terms.

To do so, attach the following notices to the program. It is safest to attach them to the start of each source file to most effectively convey the exclusion of warranty; and each file should have at least the "copyright" line and a pointer to where the full notice is found.

*one line to give the program's name and an idea of what it does.*  
*Copyright (C) yyyy name of author*

This program is free software; you can redistribute it and/or modify it under the terms of the GNU General Public License as published by the Free Software Foundation; either version 2 of the License, or (at your option) any later version.

This program is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU General Public License for more details.

You should have received a copy of the GNU General Public License along with this program; if not, write to the Free Software Foundation, Inc., 51 Franklin Street, Fifth Floor, Boston, MA 02110-1301, USA.

Also add information on how to contact you by electronic and paper mail.

If the program is interactive, make it output a short notice like this when it starts in an interactive mode:

Gnomovision version 69, Copyright (C) year name of author Gnomovision comes with ABSOLUTELY NO WARRANTY; for details type `show w'. This is free software, and you are welcome to redistribute it under certain conditions; type `show c' for details.

The hypothetical commands ``show w`` and ``show c`` should show the appropriate parts of the General Public License. Of course, the commands you use may be called something other than ``show w`` and ``show c``; they could even be mouse-clicks or menu items--whatever suits your program.

You should also get your employer (if you work as a programmer) or your school, if any, to sign a “copyright disclaimer” for the program, if necessary. Here is a sample; alter the names:

```
Yoyodyne, Inc., hereby disclaims all copyright
interest in the program `Gnomovision'
(which makes passes at compilers) written
by James Hacker.
```

```
signature of Ty Coon, 1 April 1989
Ty Coon, President of Vice
```

This General Public License does not permit incorporating your program into proprietary programs. If your program is a subroutine library, you may consider it more useful to permit linking proprietary applications with the library. If this is what you want to do, use the [GNU Lesser General Public License](#) instead of this License.

## 14.5 GNU LIBRARY GENERAL PUBLIC LICENSE

### Version 2, June 1991

Copyright (C) 1991 Free Software Foundation, Inc. 51 Franklin St, Fifth Floor, Boston, MA 02110-1301, USA

Everyone is permitted to copy and distribute verbatim copies of this license document, but changing it is not allowed.

[This is the first released version of the library GPL. It is numbered 2 because it goes with version 2 of the ordinary GPL.]

#### Preamble

The licenses for most software are designed to take away your freedom to share and change it. By contrast, the GNU General Public Licenses are intended to guarantee your freedom to share and change free software--to make sure the software is free for all its users.

This license, the Library General Public License, applies to some specially designated Free Software Foundation software, and to any other libraries whose authors decide to use it. You can use it for your libraries, too.

When we speak of free software, we are referring to freedom, not price. Our General Public Licenses are designed to make sure that you have the freedom to distribute copies of free software (and charge for this service if you wish), that you receive source code or can get it if you want it, that you can change the software or use pieces of it in new free programs; and that you know you can do these things.

To protect your rights, we need to make restrictions that forbid anyone to deny you these rights or to ask you to surrender the rights. These restrictions translate to certain responsibilities for you if you distribute copies of the library, or if you modify it.

For example, if you distribute copies of the library, whether gratis or for a fee, you must give the recipients all the rights that we gave you. You must make sure that they, too, receive or can get the source code. If you link a program with the library, you must provide complete object files to the recipients so that they can relink them with the library, after making changes to the library and recompiling it. And you must show them these terms so they know their rights.

Our method of protecting your rights has two steps: (1) copyright the library, and (2) offer you this license which gives you legal permission to copy, distribute and/or modify the library.

Also, for each distributor's protection, we want to make certain that everyone understands that there is no warranty for this free library. If the library is modified by someone else and passed on, we want its recipients to know that what they have is not the original version, so that any problems introduced by others will not reflect on the original authors' reputations.

Finally, any free program is threatened constantly by software patents. We wish to avoid the danger that companies distributing free software will individually obtain patent licenses, thus in effect transforming the program into proprietary software. To prevent this, we have made it clear that any patent must be licensed for everyone's free use or not licensed at all.

Most GNU software, including some libraries, is covered by the ordinary GNU General Public License, which was designed for utility programs. This license, the GNU Library General Public License, applies to certain designated libraries. This license is quite different from the ordinary one; be sure to read it in full, and don't assume that anything in it is the same as in the ordinary license.

The reason we have a separate public license for some libraries is that they blur the distinction we usually make between modifying or adding to a program and simply using it. Linking a program with a library, without changing the library, is in some sense simply using the library, and is analogous to running a utility program or application program. However, in a textual and legal sense, the linked executable is a combined work, a derivative of the original library, and the ordinary General Public License treats it as such.

Because of this blurred distinction, using the ordinary General Public License for libraries did not effectively promote software sharing, because most developers did not use the libraries. We concluded that weaker conditions might promote sharing better.

However, unrestricted linking of non-free programs would deprive the users of those programs of all benefit from the free status of the libraries themselves. This Library General Public License is intended to permit developers of non-free programs to use free libraries, while preserving your freedom as a user of such programs to change the free libraries that are incorporated in them. (We have not seen how to achieve this as regards changes in header files, but we have achieved it as regards changes in the actual functions of the Library.) The hope is that this will lead to faster development of free libraries.

The precise terms and conditions for copying, distribution and modification follow. Pay close attention to the difference between a "work based on the library" and a "work that uses the library". The former contains code derived from the library, while the latter only works together with the library.

Note that it is possible for a library to be covered by the ordinary General Public License rather than by this special one.

## **TERMS AND CONDITIONS FOR COPYING, DISTRIBUTION AND MODIFICATION**

**0.** This License Agreement applies to any software library which contains a notice placed by the copyright holder or other authorized party saying it may be distributed under the terms of this Library General Public License (also called "this License"). Each licensee is addressed as "you".

A "library" means a collection of software functions and/or data prepared so as to be conveniently linked with application programs (which use some of those functions and data) to form executables.

The “Library”, below, refers to any such software library or work which has been distributed under these terms. A “work based on the Library” means either the Library or any derivative work under copyright law: that is to say, a work containing the Library or a portion of it, either verbatim or with modifications and/or translated straightforwardly into another language. (Hereinafter, translation is included without limitation in the term “modification”.)

“Source code” for a work means the preferred form of the work for making modifications to it. For a library, complete source code means all the source code for all modules it contains, plus any associated interface definition files, plus the scripts used to control compilation and installation of the library.

Activities other than copying, distribution and modification are not covered by this License; they are outside its scope. The act of running a program using the Library is not restricted, and output from such a program is covered only if its contents constitute a work based on the Library (independent of the use of the Library in a tool for writing it). Whether that is true depends on what the Library does and what the program that uses the Library does.

1. You may copy and distribute verbatim copies of the Library’s complete source code as you receive it, in any medium, provided that you conspicuously and appropriately publish on each copy an appropriate copyright notice and disclaimer of warranty; keep intact all the notices that refer to this License and to the absence of any warranty; and distribute a copy of this License along with the Library.

You may charge a fee for the physical act of transferring a copy, and you may at your option offer warranty protection in exchange for a fee.

2. You may modify your copy or copies of the Library or any portion of it, thus forming a work based on the Library, and copy and distribute such modifications or work under the terms of Section 1 above, provided that you also meet all of these conditions:

- a) The modified work must itself be a software library.
- b) You must cause the files modified to carry prominent notices stating that you changed the files and the date of any change.
- c) You must cause the whole of the work to be licensed at no charge to all third parties under the terms of this License.
- d) If a facility in the modified Library refers to a function or a table of data to be supplied by an application program that uses the facility, other than as an argument passed when the facility is invoked, then you must make a good faith effort to ensure that, in the event an application does not supply such function or table, the facility still operates, and performs whatever part of its purpose remains meaningful.

(For example, a function in a library to compute square roots has a purpose that is entirely well-defined independent of the application. Therefore, Subsection 2d requires that any application-supplied function or table used by this function must be optional: if the application does not supply it, the square root function must still compute square roots.)

These requirements apply to the modified work as a whole. If identifiable sections of that work are not derived from the Library, and can be reasonably considered independent and separate works in themselves, then this License, and its terms, do not apply to those sections when you distribute them as separate works. But when you distribute the same sections as part of a whole which is a work based on the Library, the distribution of the whole must be on the terms of this License, whose permissions for other licensees extend to the entire whole, and thus to each and every part regardless of who wrote it.

Thus, it is not the intent of this section to claim rights or contest your rights to work written entirely by you; rather, the intent is to exercise the right to control the distribution of derivative or collective works based on the Library.

In addition, mere aggregation of another work not based on the Library with the Library (or with a work based on the Library) on a volume of a storage or distribution medium does not bring the other work under the scope of this License.

3. You may opt to apply the terms of the ordinary GNU General Public License instead of this License to a given copy of the Library. To do this, you must alter all the notices that refer to this License, so that they refer to the ordinary GNU General Public License, version 2, instead of to this License. (If a newer version than version 2 of the ordinary GNU General Public License has appeared, then you can specify that version instead if you wish.) Do not make any other change in these notices.

Once this change is made in a given copy, it is irreversible for that copy, so the ordinary GNU General Public License applies to all subsequent copies and derivative works made from that copy.

This option is useful when you wish to copy part of the code of the Library into a program that is not a library.

4. You may copy and distribute the Library (or a portion or derivative of it, under Section 2) in object code or executable form under the terms of Sections 1 and 2 above provided that you accompany it with the complete corresponding machine-readable source code, which must be distributed under the terms of Sections 1 and 2 above on a medium customarily used for software interchange.

If distribution of object code is made by offering access to copy from a designated place, then offering equivalent access to copy the source code from the same place satisfies the requirement to distribute the source code, even though third parties are not compelled to copy the source along with the object code.

5. A program that contains no derivative of any portion of the Library, but is designed to work with the Library by being compiled or linked with it, is called a “work that uses the Library”. Such a work, in isolation, is not a derivative work of the Library, and therefore falls outside the scope of this License.

However, linking a “work that uses the Library” with the Library creates an executable that is a derivative of the Library (because it contains portions of the Library), rather than a “work that uses the library”. The executable is therefore covered by this License. Section 6 states terms for distribution of such executables.

When a “work that uses the Library” uses material from a header file that is part of the Library, the object code for the work may be a derivative work of the Library even though the source code is not. Whether this is true is especially significant if the work can be linked without the Library, or if the work is itself a library. The threshold for this to be true is not precisely defined by law.

If such an object file uses only numerical parameters, data structure layouts and accessors, and small macros and small inline functions (ten lines or less in length), then the use of the object file is unrestricted, regardless of whether it is legally a derivative work. (Executables containing this object code plus portions of the Library will still fall under Section 6.)

Otherwise, if the work is a derivative of the Library, you may distribute the object code for the work under the terms of Section 6. Any executables containing that work also fall under Section 6, whether or not they are linked directly with the Library itself.

6. As an exception to the Sections above, you may also compile or link a “work that uses the Library” with the Library to produce a work containing portions of the Library, and distribute that work under terms of your choice, provided that the terms permit modification of the work for the customer’s own use and reverse engineering for debugging such modifications.

You must give prominent notice with each copy of the work that the Library is used in it and that the Library and its use are covered by this License. You must supply a copy of this License. If the work during execution displays copyright notices, you must include the copyright notice for the Library among them, as well as a reference directing the user to the copy of this License. Also, you must do one of these things:

- a) Accompany the work with the complete corresponding machine-readable source code for the Library including whatever changes were used in the work (which must be distributed under Sections 1 and 2 above); and, if the work is an executable linked with the Library, with the complete machine-readable “work that uses the Library”, as object code and/or source code, so that the user can modify the Library and then relink to produce a modified executable containing the modified Library. (It is understood that the user who changes the contents of definitions files in the Library will not necessarily be able to recompile the application to use the modified definitions.)
- b) Accompany the work with a written offer, valid for at least three years, to give the same user the materials specified in Subsection 6a, above, for a charge no more than the cost of performing this distribution.
- c) If distribution of the work is made by offering access to copy from a designated place, offer equivalent access to copy the above specified materials from the same place.
- d) Verify that the user has already received a copy of these materials or that you have already sent this user a copy.

For an executable, the required form of the “work that uses the Library” must include any data and utility programs needed for reproducing the executable from it. However, as a special exception, the source code distributed need not include anything that is normally distributed (in either source or binary form) with the major components (compiler, kernel, and so on) of the operating system on which the executable runs, unless that component itself accompanies the executable.

It may happen that this requirement contradicts the license restrictions of other proprietary libraries that do not normally accompany the operating system. Such a contradiction means you cannot use both them and the Library together in an executable that you distribute.

**7.** You may place library facilities that are a work based on the Library side-by-side in a single library together with other library facilities not covered by this License, and distribute such a combined library, provided that the separate distribution of the work based on the Library and of the other library facilities is otherwise permitted, and provided that you do these two things:

- a) Accompany the combined library with a copy of the same work based on the Library, uncombined with any other library facilities. This must be distributed under the terms of the Sections above.
- b) Give prominent notice with the combined library of the fact that part of it is a work based on the Library, and explaining where to find the accompanying uncombined form of the same work.

**8.** You may not copy, modify, sublicense, link with, or distribute the Library except as expressly provided under this License. Any attempt otherwise to copy, modify, sublicense, link with, or distribute the Library is void, and will automatically terminate your rights under this License. However, parties who have received copies, or rights, from you under this License will not have their licenses terminated so long as such parties remain in full compliance.

**9.** You are not required to accept this License, since you have not signed it. However, nothing else grants you permission to modify or distribute the Library or its derivative works. These actions are prohibited by law if you do not accept this License. Therefore, by modifying or distributing the Library (or any work based on the Library), you indicate your acceptance of this License to do so, and all its terms and conditions for copying, distributing or modifying the Library or works based on it.

**10.** Each time you redistribute the Library (or any work based on the Library), the recipient automatically receives a license from the original licensor to copy, distribute, link with or modify the Library subject to these terms and conditions. You may not impose any further restrictions on the recipients’ exercise of the rights granted herein. You are not responsible for enforcing compliance by third parties to this License.

11. If, as a consequence of a court judgment or allegation of patent infringement or for any other reason (not limited to patent issues), conditions are imposed on you (whether by court order, agreement or otherwise) that contradict the conditions of this License, they do not excuse you from the conditions of this License. If you cannot distribute so as to satisfy simultaneously your obligations under this License and any other pertinent obligations, then as a consequence you may not distribute the Library at all. For example, if a patent license would not permit royalty-free redistribution of the Library by all those who receive copies directly or indirectly through you, then the only way you could satisfy both it and this License would be to refrain entirely from distribution of the Library.

If any portion of this section is held invalid or unenforceable under any particular circumstance, the balance of the section is intended to apply, and the section as a whole is intended to apply in other circumstances.

It is not the purpose of this section to induce you to infringe any patents or other property right claims or to contest validity of any such claims; this section has the sole purpose of protecting the integrity of the free software distribution system which is implemented by public license practices. Many people have made generous contributions to the wide range of software distributed through that system in reliance on consistent application of that system; it is up to the author/donor to decide if he or she is willing to distribute software through any other system and a licensee cannot impose that choice.

This section is intended to make thoroughly clear what is believed to be a consequence of the rest of this License.

12. If the distribution and/or use of the Library is restricted in certain countries either by patents or by copyrighted interfaces, the original copyright holder who places the Library under this License may add an explicit geographical distribution limitation excluding those countries, so that distribution is permitted only in or among countries not thus excluded. In such case, this License incorporates the limitation as if written in the body of this License.

13. The Free Software Foundation may publish revised and/or new versions of the Library General Public License from time to time. Such new versions will be similar in spirit to the present version, but may differ in detail to address new problems or concerns.

Each version is given a distinguishing version number. If the Library specifies a version number of this License which applies to it and “any later version”, you have the option of following the terms and conditions either of that version or of any later version published by the Free Software Foundation. If the Library does not specify a license version number, you may choose any version ever published by the Free Software Foundation.

14. If you wish to incorporate parts of the Library into other free programs whose distribution conditions are incompatible with these, write to the author to ask for permission. For software which is copyrighted by the Free Software Foundation, write to the Free Software Foundation; we sometimes make exceptions for this. Our decision will be guided by the two goals of preserving the free status of all derivatives of our free software and of promoting the sharing and reuse of software generally.

## **NO WARRANTY**

15. BECAUSE THE LIBRARY IS LICENSED FREE OF CHARGE, THERE IS NO WARRANTY FOR THE LIBRARY, TO THE EXTENT PERMITTED BY APPLICABLE LAW. EXCEPT WHEN OTHERWISE STATED IN WRITING THE COPYRIGHT HOLDERS AND/OR OTHER PARTIES PROVIDE THE LIBRARY “AS IS” WITHOUT WARRANTY OF ANY KIND, EITHER EXPRESSED OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE LIBRARY IS WITH YOU. SHOULD THE LIBRARY PROVE DEFECTIVE, YOU ASSUME THE COST OF ALL NECESSARY SERVICING, REPAIR OR CORRECTION.

**16.** IN NO EVENT UNLESS REQUIRED BY APPLICABLE LAW OR AGREED TO IN WRITING WILL ANY COPYRIGHT HOLDER, OR ANY OTHER PARTY WHO MAY MODIFY AND/OR REDISTRIBUTE THE LIBRARY AS PERMITTED ABOVE, BE LIABLE TO YOU FOR DAMAGES, INCLUDING ANY GENERAL, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING OUT OF THE USE OR INABILITY TO USE THE LIBRARY (INCLUDING BUT NOT LIMITED TO LOSS OF DATA OR DATA BEING RENDERED INACCURATE OR LOSSES SUSTAINED BY YOU OR THIRD PARTIES OR A FAILURE OF THE LIBRARY TO OPERATE WITH ANY OTHER SOFTWARE), EVEN IF SUCH HOLDER OR OTHER PARTY HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES.

END OF TERMS AND CONDITIONS

### **How to Apply These Terms to Your New Libraries**

If you develop a new library, and you want it to be of the greatest possible use to the public, we recommend making it free software that everyone can redistribute and change. You can do so by permitting redistribution under these terms (or, alternatively, under the terms of the ordinary General Public License).

To apply these terms, attach the following notices to the library. It is safest to attach them to the start of each source file to most effectively convey the exclusion of warranty; and each file should have at least the “copyright” line and a pointer to where the full notice is found.

*one line to give the library's name and an idea of what it does.*  
*Copyright (C) year name of author*

This library is free software; you can redistribute it and/or modify it under the terms of the GNU Library General Public License as published by the Free Software Foundation; either version 2 of the License, or (at your option) any later version.

This library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Library General Public License for more details.

You should have received a copy of the GNU Library General Public License along with this library; if not, write to the Free Software Foundation, Inc., 51 Franklin St, Fifth Floor, Boston, MA 02110-1301, USA.

Also add information on how to contact you by electronic and paper mail.

You should also get your employer (if you work as a programmer) or your school, if any, to sign a “copyright disclaimer” for the library, if necessary. Here is a sample; alter the names:

Yoyodyne, Inc., hereby disclaims all copyright interest in the library `Frob' (a library for tweaking knobs) written by James Random Hacker.

*signature of Ty Coon, 1 April 1990*  
Ty Coon, President of Vice

That's all there is to it!

---

## 14.6 GNU LESSER GENERAL PUBLIC LICENSE

### Version 2.1, February 1999

Copyright (C) 1991, 1999 Free Software Foundation, Inc. 51 Franklin Street,  
Fifth Floor, Boston, MA 02110-1301 USA

Everyone is permitted to copy and distribute verbatim copies of this license document, but changing it is not allowed.

[This is the first released version of the Lesser GPL. It also counts as the successor of the GNU Library Public License, version 2, hence the version number 2.1.]

#### Preamble

The licenses for most software are designed to take away your freedom to share and change it. By contrast, the GNU General Public Licenses are intended to guarantee your freedom to share and change free software--to make sure the software is free for all its users.

This license, the Lesser General Public License, applies to some specially designated software packages--typically libraries--of the Free Software Foundation and other authors who decide to use it. You can use it too, but we suggest you first think carefully about whether this license or the ordinary General Public License is the better strategy to use in any particular case, based on the explanations below.

When we speak of free software, we are referring to freedom of use, not price. Our General Public Licenses are designed to make sure that you have the freedom to distribute copies of free software (and charge for this service if you wish); that you receive source code or can get it if you want it; that you can change the software and use pieces of it in new free programs; and that you are informed that you can do these things.

To protect your rights, we need to make restrictions that forbid distributors to deny you these rights or to ask you to surrender these rights. These restrictions translate to certain responsibilities for you if you distribute copies of the library or if you modify it.

For example, if you distribute copies of the library, whether gratis or for a fee, you must give the recipients all the rights that we gave you. You must make sure that they, too, receive or can get the source code. If you link other code with the library, you must provide complete object files to the recipients, so that they can relink them with the library after making changes to the library and recompiling it. And you must show them these terms so they know their rights.

We protect your rights with a two-step method: (1) we copyright the library, and (2) we offer you this license, which gives you legal permission to copy, distribute and/or modify the library.

To protect each distributor, we want to make it very clear that there is no warranty for the free library. Also, if the library is modified by someone else and passed on, the recipients should know that what they have is not the original version, so that the original author's reputation will not be affected by problems that might be introduced by others.

Finally, software patents pose a constant threat to the existence of any free program. We wish to make sure that a company cannot effectively restrict the users of a free program by obtaining a restrictive license from a patent holder. Therefore, we insist that any patent license obtained for a version of the library must be consistent with the full freedom of use specified in this license.

Most GNU software, including some libraries, is covered by the ordinary GNU General Public License. This license, the GNU Lesser General Public License, applies to certain designated libraries, and is quite different from the ordinary General Public License. We use this license for certain libraries in order to permit linking those libraries into non-free programs.

When a program is linked with a library, whether statically or using a shared library, the combination of the two is legally speaking a combined work, a derivative of the original library. The ordinary General Public License therefore permits such linking only if the entire combination fits its criteria of freedom. The Lesser General Public License permits more lax criteria for linking other code with the library.

We call this license the “Lesser” General Public License because it does Less to protect the user’s freedom than the ordinary General Public License. It also provides other free software developers Less of an advantage over competing non-free programs. These disadvantages are the reason we use the ordinary General Public License for many libraries. However, the Lesser license provides advantages in certain special circumstances.

For example, on rare occasions, there may be a special need to encourage the widest possible use of a certain library, so that it becomes a de-facto standard. To achieve this, non-free programs must be allowed to use the library. A more frequent case is that a free library does the same job as widely used non-free libraries. In this case, there is little to gain by limiting the free library to free software only, so we use the Lesser General Public License.

In other cases, permission to use a particular library in non-free programs enables a greater number of people to use a large body of free software. For example, permission to use the GNU C Library in non-free programs enables many more people to use the whole GNU operating system, as well as its variant, the GNU/Linux operating system.

Although the Lesser General Public License is Less protective of the users’ freedom, it does ensure that the user of a program that is linked with the Library has the freedom and the wherewithal to run that program using a modified version of the Library.

The precise terms and conditions for copying, distribution and modification follow. Pay close attention to the difference between a “work based on the library” and a “work that uses the library”. The former contains code derived from the library, whereas the latter must be combined with the library in order to run.

## **TERMS AND CONDITIONS FOR COPYING, DISTRIBUTION AND MODIFICATION**

**0.** This License Agreement applies to any software library or other program which contains a notice placed by the copyright holder or other authorized party saying it may be distributed under the terms of this Lesser General Public License (also called “this License”). Each licensee is addressed as “you”.

A “library” means a collection of software functions and/or data prepared so as to be conveniently linked with application programs (which use some of those functions and data) to form executables.

The “Library”, below, refers to any such software library or work which has been distributed under these terms. A “work based on the Library” means either the Library or any derivative work under copyright law: that is to say, a work containing the Library or a portion of it, either verbatim or with modifications and/or translated straightforwardly into another language. (Hereinafter, translation is included without limitation in the term “modification”.)

“Source code” for a work means the preferred form of the work for making modifications to it. For a library, complete source code means all the source code for all modules it contains, plus any associated interface definition files, plus the scripts used to control compilation and installation of the library.

Activities other than copying, distribution and modification are not covered by this License; they are outside its scope. The act of running a program using the Library is not restricted, and output from such a program is covered only if its contents constitute a work based on the Library (independent of the use of the Library in a tool for writing it). Whether that is true depends on what the Library does and what the program that uses the Library does.

**1.** You may copy and distribute verbatim copies of the Library’s complete source code as you receive it, in any medium, provided that you conspicuously and appropriately publish on each copy an appropriate copyright notice and disclaimer of warranty; keep intact all the notices that refer to this License and to the absence of any warranty; and distribute a copy of this License along with the Library.

You may charge a fee for the physical act of transferring a copy, and you may at your option offer warranty protection in exchange for a fee.

**2.** You may modify your copy or copies of the Library or any portion of it, thus forming a work based on the Library, and copy and distribute such modifications or work under the terms of Section 1 above, provided that you also meet all of these conditions:

- a) The modified work must itself be a software library.
- b) You must cause the files modified to carry prominent notices stating that you changed the files and the date of any change.
- c) You must cause the whole of the work to be licensed at no charge to all third parties under the terms of this License.
- d) If a facility in the modified Library refers to a function or a table of data to be supplied by an application program that uses the facility, other than as an argument passed when the facility is invoked, then you must make a good faith effort to ensure that, in the event an application does not supply such function or table, the facility still operates, and performs whatever part of its purpose remains meaningful.

(For example, a function in a library to compute square roots has a purpose that is entirely well-defined independent of the application. Therefore, Subsection 2d requires that any application-supplied function or table used by this function must be optional: if the application does not supply it, the square root function must still compute square roots.)

These requirements apply to the modified work as a whole. If identifiable sections of that work are not derived from the Library, and can be reasonably considered independent and separate works in themselves, then this License, and its terms, do not apply to those sections when you distribute them as separate works. But when you distribute the same sections as part of a whole which is a work based on the Library, the distribution of the whole must be on the terms of this License, whose permissions for other licensees extend to the entire whole, and thus to each and every part regardless of who wrote it.

Thus, it is not the intent of this section to claim rights or contest your rights to work written entirely by you; rather, the intent is to exercise the right to control the distribution of derivative or collective works based on the Library.

In addition, mere aggregation of another work not based on the Library with the Library (or with a work based on the Library) on a volume of a storage or distribution medium does not bring the other work under the scope of this License.

**3.** You may opt to apply the terms of the ordinary GNU General Public License instead of this License to a given copy of the Library. To do this, you must alter all the notices that refer to this License, so that they refer to the ordinary GNU General Public License, version 2, instead of to this License. (If a newer version than version 2 of the ordinary GNU General Public License has appeared, then you can specify that version instead if you wish.) Do not make any other change in these notices.

Once this change is made in a given copy, it is irreversible for that copy, so the ordinary GNU General Public License applies to all subsequent copies and derivative works made from that copy.

This option is useful when you wish to copy part of the code of the Library into a program that is not a library.

**4.** You may copy and distribute the Library (or a portion or derivative of it, under Section 2) in object code or executable form under the terms of Sections 1 and 2 above provided that you accompany it with the complete corresponding machine-readable source code, which must be distributed under the terms of Sections 1 and 2 above on a medium customarily used for software interchange.

If distribution of object code is made by offering access to copy from a designated place, then offering equivalent access to copy the source code from the same place satisfies the requirement to distribute the source code, even though third parties are not compelled to copy the source along with the object code.

5. A program that contains no derivative of any portion of the Library, but is designed to work with the Library by being compiled or linked with it, is called a “work that uses the Library”. Such a work, in isolation, is not a derivative work of the Library, and therefore falls outside the scope of this License.

However, linking a “work that uses the Library” with the Library creates an executable that is a derivative of the Library (because it contains portions of the Library), rather than a “work that uses the library”. The executable is therefore covered by this License. Section 6 states terms for distribution of such executables.

When a “work that uses the Library” uses material from a header file that is part of the Library, the object code for the work may be a derivative work of the Library even though the source code is not. Whether this is true is especially significant if the work can be linked without the Library, or if the work is itself a library. The threshold for this to be true is not precisely defined by law.

If such an object file uses only numerical parameters, data structure layouts and accessors, and small macros and small inline functions (ten lines or less in length), then the use of the object file is unrestricted, regardless of whether it is legally a derivative work. (Executables containing this object code plus portions of the Library will still fall under Section 6.)

Otherwise, if the work is a derivative of the Library, you may distribute the object code for the work under the terms of Section 6. Any executables containing that work also fall under Section 6, whether or not they are linked directly with the Library itself.

6. As an exception to the Sections above, you may also combine or link a “work that uses the Library” with the Library to produce a work containing portions of the Library, and distribute that work under terms of your choice, provided that the terms permit modification of the work for the customer’s own use and reverse engineering for debugging such modifications.

You must give prominent notice with each copy of the work that the Library is used in it and that the Library and its use are covered by this License. You must supply a copy of this License. If the work during execution displays copyright notices, you must include the copyright notice for the Library among them, as well as a reference directing the user to the copy of this License. Also, you must do one of these things:

- a) Accompany the work with the complete corresponding machine-readable source code for the Library including whatever changes were used in the work (which must be distributed under Sections 1 and 2 above); and, if the work is an executable linked with the Library, with the complete machine-readable “work that uses the Library”, as object code and/or source code, so that the user can modify the Library and then relink to produce a modified executable containing the modified Library. (It is understood that the user who changes the contents of definitions files in the Library will not necessarily be able to recompile the application to use the modified definitions.)
- b) Use a suitable shared library mechanism for linking with the Library. A suitable mechanism is one that (1) uses at run time a copy of the library already present on the user’s computer system, rather than copying library functions into the executable, and (2) will operate properly with a modified version of the library, if the user installs one, as long as the modified version is interface-compatible with the version that the work was made with.
- c) Accompany the work with a written offer, valid for at least three years, to give the same user the materials specified in Subsection 6a, above, for a charge no more than the cost of performing this distribution.
- d) If distribution of the work is made by offering access to copy from a designated place, offer equivalent access to copy the above specified materials from the same place.
- e) Verify that the user has already received a copy of these materials or that you have already sent this user a copy.

For an executable, the required form of the “work that uses the Library” must include any data and utility programs needed for reproducing the executable from it. However, as a special exception, the materials to be distributed need not include anything that is normally distributed (in either source or binary form) with the major components (compiler, kernel, and so on) of the operating system on which the executable runs, unless that component itself accompanies the executable.

It may happen that this requirement contradicts the license restrictions of other proprietary libraries that do not normally accompany the operating system. Such a contradiction means you cannot use both them and the Library together in an executable that you distribute.

**7.** You may place library facilities that are a work based on the Library side-by-side in a single library together with other library facilities not covered by this License, and distribute such a combined library, provided that the separate distribution of the work based on the Library and of the other library facilities is otherwise permitted, and provided that you do these two things:

- a) Accompany the combined library with a copy of the same work based on the Library, uncombined with any other library facilities. This must be distributed under the terms of the Sections above.
- b) Give prominent notice with the combined library of the fact that part of it is a work based on the Library, and explaining where to find the accompanying uncombined form of the same work.

**8.** You may not copy, modify, sublicense, link with, or distribute the Library except as expressly provided under this License. Any attempt otherwise to copy, modify, sublicense, link with, or distribute the Library is void, and will automatically terminate your rights under this License. However, parties who have received copies, or rights, from you under this License will not have their licenses terminated so long as such parties remain in full compliance.

9. You are not required to accept this License, since you have not signed it. However, nothing else grants you permission to modify or distribute the Library or its derivative works. These actions are prohibited by law if you do not accept this License. Therefore, by modifying or distributing the Library (or any work based on the Library), you indicate your acceptance of this License to do so, and all its terms and conditions for copying, distributing or modifying the Library or works based on it.

10. Each time you redistribute the Library (or any work based on the Library), the recipient automatically receives a license from the original licensor to copy, distribute, link with or modify the Library subject to these terms and conditions. You may not impose any further restrictions on the recipients' exercise of the rights granted herein. You are not responsible for enforcing compliance by third parties with this License.

11. If, as a consequence of a court judgment or allegation of patent infringement or for any other reason (not limited to patent issues), conditions are imposed on you (whether by court order, agreement or otherwise) that contradict the conditions of this License, they do not excuse you from the conditions of this License. If you cannot distribute so as to satisfy simultaneously your obligations under this License and any other pertinent obligations, then as a consequence you may not distribute the Library at all. For example, if a patent license would not permit royalty-free redistribution of the Library by all those who receive copies directly or indirectly through you, then the only way you could satisfy both it and this License would be to refrain entirely from distribution of the Library.

If any portion of this section is held invalid or unenforceable under any particular circumstance, the balance of the section is intended to apply, and the section as a whole is intended to apply in other circumstances.

It is not the purpose of this section to induce you to infringe any patents or other property right claims or to contest validity of any such claims; this section has the sole purpose of protecting the integrity of the free software distribution system which is implemented by public license practices. Many people have made generous contributions to the wide range of software distributed through that system in reliance on consistent application of that system; it is up to the author/donor to decide if he or she is willing to distribute software through any other system and a licensee cannot impose that choice.

This section is intended to make thoroughly clear what is believed to be a consequence of the rest of this License.

12. If the distribution and/or use of the Library is restricted in certain countries either by patents or by copyrighted interfaces, the original copyright holder who places the Library under this License may add an explicit geographical distribution limitation excluding those countries, so that distribution is permitted only in or among countries not thus excluded. In such case, this License incorporates the limitation as if written in the body of this License.

13. The Free Software Foundation may publish revised and/or new versions of the Lesser General Public License from time to time. Such new versions will be similar in spirit to the present version, but may differ in detail to address new problems or concerns.

Each version is given a distinguishing version number. If the Library specifies a version number of this License which applies to it and "any later version", you have the option of following the terms and conditions either of that version or of any later version published by the Free Software Foundation. If the Library does not specify a license version number, you may choose any version ever published by the Free Software Foundation.

14. If you wish to incorporate parts of the Library into other free programs whose distribution conditions are incompatible with these, write to the author to ask for permission. For software which is copyrighted by the Free Software Foundation, write to the Free Software Foundation; we sometimes make exceptions for this. Our decision will be guided by the two goals of preserving the free status of all derivatives of our free software and of promoting the sharing and reuse of software generally.

**NO WARRANTY**

15. BECAUSE THE LIBRARY IS LICENSED FREE OF CHARGE, THERE IS NO WARRANTY FOR THE LIBRARY, TO THE EXTENT PERMITTED BY APPLICABLE LAW. EXCEPT WHEN OTHERWISE STATED IN WRITING THE COPYRIGHT HOLDERS AND/OR OTHER PARTIES PROVIDE THE LIBRARY "AS IS" WITHOUT WARRANTY OF ANY KIND, EITHER EXPRESSED OR IMPLIED, INCLUDING, BUT NOT LIMITED TO, THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. THE ENTIRE RISK AS TO THE QUALITY AND PERFORMANCE OF THE LIBRARY IS WITH YOU. SHOULD THE LIBRARY PROVE DEFECTIVE, YOU ASSUME THE COST OF ALL NECESSARY SERVICING, REPAIR OR CORRECTION.

16. IN NO EVENT UNLESS REQUIRED BY APPLICABLE LAW OR AGREED TO IN WRITING WILL ANY COPYRIGHT HOLDER, OR ANY OTHER PARTY WHO MAY MODIFY AND/OR REDISTRIBUTE THE LIBRARY AS PERMITTED ABOVE, BE LIABLE TO YOU FOR DAMAGES, INCLUDING ANY GENERAL, SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES ARISING OUT OF THE USE OR INABILITY TO USE THE LIBRARY (INCLUDING BUT NOT LIMITED TO LOSS OF DATA OR DATA BEING RENDERED INACCURATE OR LOSSES SUSTAINED BY YOU OR THIRD PARTIES OR A FAILURE OF THE LIBRARY TO OPERATE WITH ANY OTHER SOFTWARE), EVEN IF SUCH HOLDER OR OTHER PARTY HAS BEEN ADVISED OF THE POSSIBILITY OF SUCH DAMAGES.

END OF TERMS AND CONDITIONS

**How to Apply These Terms to Your New Libraries**

If you develop a new library, and you want it to be of the greatest possible use to the public, we recommend making it free software that everyone can redistribute and change. You can do so by permitting redistribution under these terms (or, alternatively, under the terms of the ordinary General Public License).

To apply these terms, attach the following notices to the library. It is safest to attach them to the start of each source file to most effectively convey the exclusion of warranty; and each file should have at least the "copyright" line and a pointer to where the full notice is found.

*one line to give the library's name and an idea of what it does.*  
 Copyright (C) year name of author

This library is free software; you can redistribute it and/or modify it under the terms of the GNU Lesser General Public License as published by the Free Software Foundation; either version 2.1 of the License, or (at your option) any later version.

This library is distributed in the hope that it will be useful, but WITHOUT ANY WARRANTY; without even the implied warranty of MERCHANTABILITY or FITNESS FOR A PARTICULAR PURPOSE. See the GNU Lesser General Public License for more details.

You should have received a copy of the GNU Lesser General Public License along with this library; if not, write to the Free Software Foundation, Inc., 51 Franklin Street, Fifth Floor, Boston, MA 02110-1301 USA

Also add information on how to contact you by electronic and paper mail.

You should also get your employer (if you work as a programmer) or your school, if any, to sign a “copyright disclaimer” for the library, if necessary. Here is a sample; alter the names:

Yoyodyne, Inc., hereby disclaims all copyright interest in the library ‘Frob’ (a library for tweaking knobs) written by James Random Hacker.

*signature of Ty Coon*, 1 April 1990  
Ty Coon, President of Vice

That’s all there is to it!

## 14.7 Fresenius Kabi software license Version 1.0, Juni 2017

BY DOWNLOADING, INSTALLING, COPYING, ACCESSING OR USING THIS SOFTWARE, YOU AGREE TO THE TERMS OF THIS SOFTWARE LICENSE AGREEMENT (“LICENSE AGREEMENT”). IF YOU ARE ACCEPTING THESE TERMS ON BEHALF OF ANOTHER PERSON OR COMPANY OR OTHER LEGAL ENTITY, YOU REPRESENT AND WARRANT THAT YOU HAVE FULL AUTHORITY TO BIND THAT PERSON, COMPANY OR OTHER LEGAL ENTITY TO THESE TERMS.

IF YOU DO NOT AGREE TO THESE TERMS DO NOT DOWNLOAD, INSTALL, COPY, ACCESS OR USE THIS SOFTWARE, AND PROMPTLY RETURN THIS SOFTWARE AND PROOF OF ENTITLEMENT TO THE PARTY FROM WHOM YOU ACQUIRED THEM.

This software license agreement (“License Agreement”) is a legal agreement between the natural person, company, or legal entity by or on behalf of which the Software is downloaded, installed, copied, accessed, or used (any such person, company or legal entity the “Customer”), and Fresenius Kabi AG, Else-Kröner-Straße 1, 61352 Bad Homburg, Germany (“Fresenius Kabi”) governing Customer’s use of Fresenius Kabi’s proprietary CompoMat G5 *Plus* and CompoMaster Net G5 software applications (Order numbers: CompoMat G5 *Plus* 9026501, CompoMaster Net G5 9026541) and its associated user documentation (“Documentation”) (such software and Documentation are collectively referred to herein as the “Software”).

The provisions of this License Agreement only apply to the proprietary components of the Software and explicitly do not apply to any components listed in the Documentation as third party or non-proprietary software being provided under alternative licenses.

### 1. OVERVIEW.

**1.1 Orders.** Fresenius Kabi or its affiliated companies or distributors, as the case may be, offer the Software and/or related equipment such as Fresenius Kabi’s CompoMat® G5 and other blood processing equipment (the “Equipment”) for purchase by the Customer. Customer may from time to time execute purchase agreements or order forms for Software and/or Equipment (each, an “Order”). As applicable to purchases of licenses to the Software, applicable commercial terms with respect to quantity, term, pricing, payment, and the types and numbers of permitted users, sites, machines, user logins or IDs, and/or other applicable usage parameters (collectively, “Usage Parameters”) will be set forth in the applicable Order. Each Order shall incorporate and be subject to this License Agreement as it relates to the Software.

**1.2 License Key.** Use of the Software may require that Customer enters a product activation code (“License Key”) into the Software. License Keys may not be copied, modified, distributed, published, transferred or disclosed outside Customer, nor may any License Key be used on any copy of the Software or on any other machine other than the copy or machine for which it has been supplied by Fresenius Kabi. License Keys shall be deemed part of the Software for all purposes of this License Agreement.

## **2. SCOPE OF LICENSE; RESTRICTIONS.**

**2.1 License.** Subject to Customer’s compliance with the terms of this License Agreement, and for so long as this License Agreement remains in effect, Fresenius Kabi hereby grants to Customer (and not to any third party) a limited, non-exclusive, revocable, non-sublicenseable, non-transferable (except as set forth herein) license to install and use the Software solely for its intended purpose as described in the Documentation, in accordance with the Usage Parameters specified in the applicable Order, and in conjunction with Customer’s use of Fresenius Kabi’s CompoMat®G5 system. Customer may make a single copy of the Software solely for backup and archival purposes.

**2.2 Restrictions.** Customer acknowledges and agrees that Customer will not: (a) duplicate, disclose, rent, lease, lend, sublicense, sell, encumber, otherwise make the Software available to third parties, or provide commercial services based upon use of the Software; (b) modify, distribute, publish, adapt, translate, prepare derivative works from, decompile, reverse engineer, disassemble or otherwise attempt to derive source code from the Software, except to the extent expressly permitted by applicable law notwithstanding this limitation; (c) attempt to circumvent or disable any security or usage limitation features of the Software by any means or in any manner; (d) remove, obscure, or alter any copyright, trademark or other proprietary rights, notices or legends affixed to or contained within the Software; (e) use any automated or manual process to copy material from the Software except as permitted by the Documentation; or (g) use the Software in any manner in violation of applicable law. Customer may not transfer or assign this License Agreement, any license rights granted herein, nor the Software or any component thereof, except as expressly permitted herein. Customer may not separate its component parts for distribution or any other use.

**2.3 OWNERSHIP.** The Software is licensed, not sold, and is protected by copyright and other intellectual property laws and treaties. Fresenius Kabi and/or its licensors own all rights, title, and interest in the Software, including, without limitation, all copyrights, trade secrets, trademarks, patent rights, ideas, concepts, know-how, techniques, inventions, discoveries, improvements, modifications, derivative works (including, without limitation, any related modifications, enhancements, derivative works, upgrades, updates or bug fixes made by Fresenius Kabi or its licensors whether or not based upon feedback, input or direction provided by Customer), and other intellectual property and proprietary rights therein. Except for the limited license rights granted hereunder, Customer has no rights in or to the Software or Documentation or any copies thereof, except to possess and use them in accordance with this License Agreement. To the extent Customer is permitted to make copies hereunder, Customer shall include Fresenius Kabi’s copyright, trademark or other proprietary rights notices or legends on any complete or partial copies of the Software in the same form and location as the notice appears on the original work. Fresenius Kabi reserves all rights not expressly granted to Customer in this License Agreement and Customer agrees not to take any action that interferes, in any manner, with Fresenius Kabi’s or its licensors’ rights with respect to the Software.

**3. CONFIDENTIALITY.** The Software contains and constitutes valuable confidential and trade secret information of Fresenius Kabi, including without limitation, data processing algorithms, innovations and concepts (“Confidential Information”). Customer agrees not to disclose nor permit access to the Software (including, for the avoidance of doubt, the Documentation) to any third parties not under an obligation of confidentiality to Customer to protect the confidentiality of information Customer discloses to them, including the Software, nor to utilize for Customer’s own benefit any such Confidential Information except in connection with Customer’s permitted use thereof. This obligation shall survive the termination of this License Agreement and any Order.

**4. UPDATES.** Fresenius Kabi may, at its option and subject to a valid maintenance and service agreement with Customer, periodically make available updates, corrections, patches or fixes addressing operational or other issues related to the Software (collectively, “Updates”). Updates shall be deemed part of the Software.

**5. CUSTOMER USERS.** Customer is responsible for all activity of Customer's employees, agents, representatives and others accessing or using the Software through or on behalf of Customer. Customer is also responsible for (i) identifying individuals who Customer determines should have access to the Software ("Users"); (ii) monitoring Users' access to and use of the Software; (iii) acting upon any suspected or unauthorized access to and use of Software; and (iv) ensuring each User's compliance with the License Agreement.

**6. LIMITED WARRANTIES.**

**6.1 Warranty.** Fresenius Kabi warrants, for Customer's benefit alone, that for a period of twelve (12) months from delivery of the Software initially supplied to Customer (the "Warranty Period"): (a) the media on which the Software is furnished will be free from defects in materials and workmanship under normal use, and (b) the Software's performance will substantially conform to descriptions thereof contained in Fresenius Kabi's Documentation.

**6.2 Exclusive Remedy.** Fresenius Kabi's entire liability and Customer's sole and exclusive remedy for a breach of the foregoing warranties shall be: (a) in the case of defects in media in breach of the warranty set forth in Section 7.1(a), the replacement of such media by Fresenius Kabi, and (b) in the case of defects in the Software in breach of the warranty set forth in Section 7.1(b), Fresenius Kabi shall use commercially reasonable efforts to correct any such defect or, at Fresenius Kabi option, replace the defective Software or terminate this License Agreement and refund the applicable license fees paid upon receipt of all copies of the Software in Customer's possession and Customer's written certification that no additional copies are in Customer's possession or (at Fresenius Kabi election) that Customer has deleted all such copies. To receive the foregoing remedies, Customer must report any alleged breach of warranty to Fresenius Kabi in writing promptly after discovery and in reasonable detail. The warranties set forth in Section 7.1 do not cover any copy of the Software which has been altered, misused or damaged in any way by Customer or any third party, nor is Fresenius Kabi responsible for problems caused by changes in, or modifications to, any computer hardware or operating system for which the Software has been procured or which occur as a result of use of the Software in conjunction with software of third parties or with hardware which is incompatible with the Software or the operating system for which the Software has been procured.

**6.3 Warranty Limitations.** EXCEPT AS EXPRESSLY SET FORTH IN SECTION 7.1, ABOVE, NEITHER FRESENIUS KABI NOR ITS AFFILIATES OR THEIR RESPECTIVE OFFICERS, DIRECTORS, EMPLOYEES, SUBSIDIARIES, CONTRACTORS, SUBCONTRACTORS, SUPPLIERS, LICENSORS, AGENTS, PARTNERS, RESELLERS OR DISTRIBUTORS, MAKE ANY EXPRESS OR IMPLIED WARRANTIES OR REPRESENTATIONS TO CUSTOMER, ANY OF CUSTOMER'S AFFILIATES, OR TO ANY OTHER PARTY WITH RESPECT TO THE SOFTWARE, THE DOCUMENTATION, ANY EQUIPMENT OR SERVICES RELATED TO THE SOFTWARE OR OTHERWISE REGARDING THIS LICENSE AGREEMENT, WHETHER ORAL OR WRITTEN, EXPRESS, IMPLIED, OR STATUTORY. WITHOUT LIMITING THE FOREGOING, ANY IMPLIED WARRANTY OR CONDITION OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE IS EXPRESSLY EXCLUDED AND DISCLAIMED. FRESENIUS KABI DOES NOT WARRANT THAT THE SOFTWARE WILL MEET CUSTOMER'S REQUIREMENTS, THAT THE SOFTWARE IS WITHOUT DEFECT OR ERROR OR THAT THE SOFTWARE WILL OPERATE WITH ANY HARDWARE OR SOFTWARE NOT SPECIFIED IN THE DOCUMENTATION. CUSTOMER SHOULD NOT SUBSTITUTE THE SOFTWARE OR RESULTS THEREFROM FOR ITS OR ITS EMPLOYEES INDEPENDENT JUDGMENT. THE RESULTS PRODUCED BY THE SOFTWARE SHOULD NOT BE USED FOR MEDICAL DECISION MAKING, AND OUTPUTS DO NOT SUPERSEDE ANY FRESENIUS KABI PRODUCT LABELING, DIRECTIONS FOR USE, OR ANY REGULATORY GUIDELINES. FRESENIUS KABI DOES NOT ASSUME ANY LIABILITY FOR LOSSES INCURRED BY ANY USE OF OR RELIANCE UPON THE SOFTWARE PROGRAM OR USER DOCUMENTATION. CUSTOMER ASSUMES ALL RESPONSIBILITY AND RISK FOR THE INSTALLATION, USE, PERFORMANCE OF AND RESULTS OBTAINED FROM THE SOFTWARE.

**7. TERM AND TERMINATION.** This License Agreement shall remain in effect for the time period specified in the relevant Order. In absence of such specification the License Agreement shall remain in effect from the effective date of the relevant Order until its termination by either one of the Parties. Statutory rights to terminate for cause might be applicable.

## 8. INDEMNIFICATION.

**8.1 Indemnification by Fresenius Kabi.** Fresenius Kabi shall indemnify, defend, and hold harmless Customer and its respective officers, directors, representatives and agents from and against any and all losses, expenses, costs, liabilities, damages or settlement thereof (including reasonable attorneys' and expert witness' fees) that are caused by or in any manner arise out of or result from any third party claim, demand, suit or proceeding against Customer, or any of its respective officers, directors, representatives or agents for infringement or misappropriation of such third party's intellectual property rights arising from the Software. Fresenius Kabi indemnification or misappropriation obligation will not apply to the extent that the infringement action is based upon: (i) the use of the Software in combination with any product, service or activity (or any part thereof) not furnished, performed, recommended in writing, or approved in writing by Fresenius Kabi; (ii) the use of the Software in violation of this License Agreement; (iii) the use of the Software not updated to the latest version offered by Fresenius Kabi, where the latest version incorporates modifications that, in Fresenius Kabi opinion, avoid the infringement claim; or (iv) third party content supplied or transmitted by Customer or its Users. In the event a court of competent jurisdiction determines that the Software infringes or misappropriates any third party intellectual property rights, Customer shall comply with any instruction from Fresenius Kabi to discontinue, limit or otherwise modify the use of the Software including, without limitation, any component thereof. Fresenius Kabi shall at its expense: (i) procure the right for Customer to continue using the infringing Software, or (ii) promptly replace the infringing Software with a non-infringing version of equivalent function, features, characteristics and performance or modify the Software to be non-infringing without detracting from function or performance, or (iii) if the prior remedies set forth in (i) and (ii) of this Section 9.1 are commercially impractical, recede from or terminate this License Agreement. Customer shall reasonably cooperate with Fresenius Kabi, at Fresenius Kabi's expense, in defending or settling such claim. The remedies set forth herein are Customer's sole remedy with respect to claims for infringement or misappropriation.

**8.2 Indemnification by Customer.** Customer shall indemnify, defend, and hold harmless Customer and its respective officers, directors, representatives and agents from and against any and all losses, expenses, costs, liabilities, damages or settlement thereof (including reasonable attorneys' and expert witness' fees) that are caused by or in any manner arise out of or result from any third party claim, demand, suit or proceeding against Fresenius Kabi for infringement or misappropriation of any third party's intellectual property rights arising from (i) Customer's or its employees', officers', directors', representatives' or agents' use of the Software in a manner which violates any terms of this License Agreement, and/or (ii) any derivative work, modification or combination of the Software made by Customer, to the extent that the claim for infringement or misappropriation is based upon such derivative work, modification or combination, and/or (iii) the negligence, intentional misconduct or violation of applicable law by or of Customer, its Affiliates and their respective directors, officers, employees and agents. Fresenius Kabi shall reasonably cooperate with Customer, at Customer's expense, in defending or settling such claim. The remedies set forth herein are Fresenius Kabi's sole remedy with respect to claims for infringement or misappropriation.

**8.3 Claims for Indemnification.** The obligation to provide indemnification under this License Agreement shall be contingent upon the party seeking indemnification (i) providing the indemnifying party with prompt written notice of any claim for which indemnification is sought; (ii) allowing the indemnifying party to control the defense and settlement of such claim; and (iii) cooperating fully with the indemnifying party in connection with such defense and settlement. Notwithstanding the foregoing, the indemnifying party agrees not to enter into any settlement or compromise of any claim or action in a manner that admits fault or imposes any restrictions or obligations on an indemnified party without that indemnified party's prior written consent which will not be unreasonably withheld.

## 9. LIMITATION OF LIABILITY; DISCLAIMER OF DAMAGES.

**9.1 Limitation of liability.** Fresenius Kabi shall be liable under the terms of this License Agreement in accordance with the provisions set out under (a) to (e):

- a) Fresenius Kabi's liability for losses caused intentionally or with gross negligence by Fresenius Kabi, its legal representatives or senior executives and for losses caused intentionally by other assistants in performance shall not be restricted; in respect of gross negligence of other assistants in performance Fresenius Kabi's liability shall be as set forth in (e) below.
- b) Fresenius Kabi liability for death, personal injury or damage to health caused by the intent or negligence of Fresenius Kabi, its legal representatives or assistants in performance is not restricted.
- c) Fresenius Kabi shall be liable for losses arising from the lack of any warranted characteristics up to the amount which is covered by the purpose of the warranty and which was foreseeable by Fresenius Kabi at the time the warranty was given.
- d) Fresenius Kabi shall be liable in accordance with the German Product Liability Act in the event of product liability.
- e) Fresenius Kabi shall be liable for losses caused by the breach of its primary contractual obligations. Primary contractual obligations are such basic duties which form the essence of the License Agreement, which were decisive for the conclusion of the License Agreement and on the performance of which Customer may rely. If Fresenius Kabi breaches its primary obligations through simple negligence, then its ensuing liability shall be limited to the amount which was foreseeable by Fresenius Kabi at the time the respective service was performed.
- f) Fresenius Kabi shall be liable for loss of data only up to the amount of typical recovery costs which would have arisen had proper and regular data backup measures been taken by Customer.

Any more extensive liability of Fresenius Kabi is excluded on the merits.

**9.2 Basis of Agreement.** THE FOREGOING LIMITATIONS, EXCLUSIONS AND DISCLAIMERS SHALL APPLY TO THE MAXIMUM EXTENT PERMITTED BY APPLICABLE LAW, EVEN IF A STATED REMEDY FAILS ITS ESSENTIAL PURPOSE, AND CONSTITUTE A FUNDAMENTAL ELEMENT OF THE BASIS OF THE BARGAIN OF THE PARTIES AND, BUT FOR SUCH LIMITATIONS, EXCLUSIONS AND DISCLAIMERS, FRESENIUS KABI WOULD NOT BE WILLING TO ENTER INTO THIS LICENSE AGREEMENT.

**10. FOREIGN EXPORT.** Customer may not export the Software, directly or indirectly, into any country in violation of any applicable laws or regulations.

## **11. DATA.**

11.1 Throughout the Term, Customer agrees to provide Fresenius Kabi with access to the Software or Customer's computer(s) on which the Software is installed for maintenance and service purposes. Fresenius Kabi may have access to the information that may be stored in the data files of the Software or created using the Software, including data related to use of Fresenius Kabi's CompoMat®G5 or other processing devices technology (collectively, "Data"). Access to Data will allow Fresenius Kabi to provide further support, analysis and reports to Customer. It is understood and agreed by both Customer and Fresenius Kabi that all Data is not "Protected Health Information" (as such term is defined pursuant to 45 C.F.R. § 160.103); however, each Party agrees to treat the Data as confidential information.

11.2 Notwithstanding the foregoing, Fresenius Kabi is expressly permitted to aggregate and remove any Customer or individual identifiers from the Data (“De-Identified Data”). Such De-Identified Data will be considered derivative works or improvements created by Fresenius Kabi, and, as such, constitutes Fresenius Kabi’s intellectual property. To the extent Customer has any right, title or interest in or to any De-Identified Data, Customer hereby assigns, and agrees to assign, all rights, title and interests, including all intellectual property and proprietary rights, in and to such De-Identified Data to Fresenius Kabi. To the extent that any De-Identified Data does not constitute Fresenius Kabi intellectual property or cannot be so assigned to Fresenius Kabi, Customer hereby grants Fresenius Kabi and its affiliates a nonexclusive, royalty free, perpetual, irrevocable, worldwide, and sub-licensable right and license to aggregate, compile, decompile, manipulate, reproduce, modify, supplement, adapt, translate, create derivative works from, distribute, publish, disclose and otherwise use De-Identified Data for all purposes, including commercial purposes and other purposes. Upon any termination or expiration of this License Agreement, Fresenius Kabi will not be obligated to return to Customer any De-identified Data.

11.3 Customer is solely responsible for creating, validating and loading into the Software the specific Data Customer intends to use with the Software. Customer agrees that Customer will use best practices and protocols in the creation of such Data and will provide to Customer’s personnel reasonable and proper supervision, control and management of the use of the Software and Data. Customer agrees to implement commercially reasonable data security, disaster recovery, data backup and related procedures and practices for the protection of Data consistent with best practices and protocols applicable to the management of information technology and data in a healthcare environment.

**12. U.S. GOVERNMENT RESTRICTED RIGHTS.** The Software and Documentation are “Commercial Items”, as that term is defined in 48 C.F.R. §2.101, consisting of “Commercial Computer Software” and “Commercial Computer Software Documentation”, as such terms are used in 48 C.F.R. §12.212 or 48 C.F.R. §227.7202, as applicable, and are licensed with restricted rights. Consistent with 48 C.F.R. §12.212 or 48 C.F.R. §227.7202-1 through 227.7202-4, as applicable, the Commercial Computer Software and Commercial Computer Software Documentation, if licensed to the U.S. Government, shall be licensed to U.S. Government end users (a) only as Commercial Items and (b) with only those rights as are granted to all other end users pursuant to the terms and conditions herein. Unpublished - rights are reserved under the copyright laws of the United States. Manufacturer is: Fresenius Kabi AG, Else-Kröner-Straße 1, 61352 Bad Homburg, Germany.

### **13. GENERAL PROVISIONS.**

**13.1 Assignment.** Customer may not assign, sublicense, or otherwise transfer this License Agreement or the license rights granted to Customer herein, or any of Customer’s rights or obligations under this License Agreement, to any party without the prior written consent of Fresenius Kabi; provided, however, that Customer may assign this License Agreement without Fresenius Kabi’s consent in the event of a sale of all or substantially all of Customer’s assets or in the event of a merger, corporate reorganization or business consolidation of Customer’s entity. This License Agreement shall be binding upon and inure to the benefit of the parties, their legal representatives and permitted transferees, successors and assigns as permitted by this License Agreement.

**13.2 License Compliance.** To ensure that Customer’s installation and deployment of the Software is consistent with the Software license rights granted hereunder and complies with the terms of this License Agreement, Fresenius Kabi or its representatives may conduct an audit of Customer’s Software installation/deployment not more than once per year on thirty (30) days prior written notice. Such audit will require Customer to provide an unedited accurate report of all Software installed by Customer and all valid purchase documentation for all Software within thirty (30) days of request. If the audit findings demonstrate non-compliance with the Software license rights granted hereunder, Customer shall purchase the necessary licenses within fifteen (15) days of being so notified. If underpaid fees are in excess of ten percent (10%) of the value of the fees paid under this License Agreement, then Customer shall pay such underpaid fees and Fresenius Kabi’s reasonable costs of conducting the audit. This section 14.2 shall survive termination or expiration of the License Agreement for a period of two (2) years. The foregoing shall not in any way be construed to limit any other rights and/or remedies that Fresenius Kabi may have under law or equity.

- 13.3 Governing Law and Venue.** This License Agreement, and any dispute that may occur under or related to this License Agreement, shall be governed by and construed in accordance with the laws of Germany without regard to conflict of law principles and the United Nations Convention on Contracts for the International Sale of Goods. To the extent legally permissible the courts of Frankfurt/Main, Germany shall have exclusive jurisdiction regarding all disputes arising out of or in connection with this License Agreement or its validity.
- 13.4 No Waiver.** No delay or failure in exercising any right hereunder and no partial or single exercise thereof shall be deemed to constitute a waiver of such right or any other rights hereunder. No consent to a breach of any express or implied term of this License Agreement shall constitute a consent to any prior or subsequent breach.
- 13.5 Notices.** With the exception of routine administrative communications between the parties which may be sent via e-mail, all notices under this License Agreement shall be in writing to each party at their respective addresses set forth in the Order, to the attention of any individuals identified therein, with a copy to each party's respective "Legal Department." Notices shall be given personally or sent by traceable postal service mail, e.g., certified or registered mail, or private overnight courier (e.g., FedEx, UPS or DHL) and shall be deemed given upon delivery, if given personally, or when deposited with the overnight courier or national postal service with the proper postage affixed, if sent by mail, in accordance with this provision. The parties may change the address for notice by compliance with this Section.
- 13.6 Survival.** The following provisions shall survive any termination or expiration of this License Agreement: 2.2, 3, 4, 7.3, 8, 9, 10, 11, 12, 13 and 14.
- 13.7 Severability.** If any provision hereof is declared invalid by a court of competent jurisdiction, such provision shall be ineffective only to the extent of such invalidity, so that the remainder of that provision and all remaining provisions of this License Agreement shall be valid and enforceable to the fullest extent permitted by applicable law.
- 13.8 Third Party Software.** Third party materials, e.g., software, and services included in or accessed through the Software may be subject to other terms and conditions typically found in a separate license agreement, terms of use or read me file located within or near such materials and services or included within the Software. Customer is solely responsible for compliance with any applicable licenses, service agreements or terms and conditions governing the use of third party materials.
- 13.9 Force Majeure.** Notwithstanding anything in this License Agreement to the contrary, neither party will be liable for any delay or failure with respect to any obligation hereunder, nor for any damages or loss of any kind, resulting from causes beyond its reasonable control, including without limitation, acts of god, earthquake, storms or other elements of nature, labor disputes, blockages, embargoes, riots or other industrial disturbances, electrical, telecommunications, Internet or other third party supplier delay or failure, acts or orders of government, criminal acts, war or terrorism, including cyber-attack or other malicious intrusion into or breach of computer systems.
- 13.10 No Other Rights.** No rights or licenses, express or implied, are granted to either Party by this License Agreement to use in any manner any trade name or trademark of the other Party, or any other intellectual property that is not expressly covered by this License Agreement.
- 13.11 No Third Party Benefit.** The provisions of this License Agreement are for the sole benefit of the parties hereto. Except as expressly provided herein, this License Agreement neither confers any rights, benefits, or claims upon any person or entity not a party hereto nor precludes any actions against, or rights of recovery from, any persons or entities not parties hereto.
- 13.12 Conflicts.** In the event of any conflict between the terms and conditions of this License Agreement and an Order, the terms and conditions of this License Agreement shall prevail and control as such term relates to the Software.

\* \* \*