

TRANSFUSION TECHNOLOGY

Compocouch

THE COMFORTABLE DONATION
AND APHERESIS COUCHES



**FRESENIUS
KABI**

caring for life

Compocouch

The comfortable donation and apheresis couches

Quality and Comfort

Comfortable relaxation when sitting due to the twin-belt system and exchangeable bolster pad made of highly absorbent foam material.

The arm rests can be moved in and out to adjust seat width to suit individual requirements. Especially for whole blood donation the wedge shaped arm support promotes a comfortable vena puncture resulting in optimal blood flow.

The extra long foot section ensures a comfortable rest even for tall donors.

Safety

Adjustment/control buttons are arranged logically on the remote control. No tipping over of the couch in case of a donor collapse (up to 200 kg). Press only one button to immediately reach shock position. A new key system guarantees that the personnel stays in full control of the Compocouch.

Hygiene

The smooth surface is easy to clean. The arm rests are fully padded for sturdiness and high resistance. In general all seals are closed to prevent any dirt collection while not in use.



- More comfort
- Improved safety
- New design

Compocouch

Couches for comfortable donor care

Design

Fresh new colors combined with a wide range of models and functions for high variability.

Optional

Adjustable foot rest.

Accessories

A wide variety of useful accessories is available.

Maximum Load

200 kg - "Evolution" and "Start"

150 kg - "Compochair"



Compocouch models & accessories

Evolution A3/A4 The high-end model

High-end model Evolution A3/A4 with outstanding features. The optimal solution for blood donation in transfusion medicine. A4 is delivered with four, A3 with three motors (without height adjustment). The new comfort button moves back-, seat- and foot part simultaneously (rocking chair function). The angle of the arm rest is adjustable.

Start A2/A3 The economical model

The Start models have the same construction features as the Evolution models. Through simplified basic chassis and design, an improved price/benefit relation can be achieved. Seat part and arm rest angle are always fixed. Start A3 allows height adjustment.

Compochair The compact model

This model is characterized by being particularly inexpensive. The ideal choice whenever investments have to be made with a reduced budget, but high-quality product features are required. Standard version is without foot rest and neck roll.

Paper roll holder

The paper source for a hygienic covering of the couches.

Safe and comfortable remote control


The remote control can be locked and unlocked via a magnetic key. The green colored comfort key moves all parts simultaneously like a rocking chair.

Rechargeable battery

A rechargeable battery is available for the Compocouch to support safety. In case of a power breakdown the Compocouch can fully operate on battery (option not available for Compochair).



A wide variety of attractive and useful accessories is available for the Compocouch models

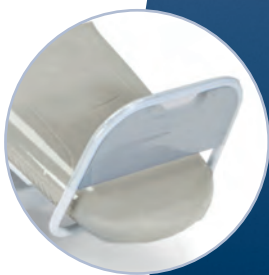

Technical data	Start A2	Start A3	Evolution A3	Evolution A4	Compochair
Width of arm rest	13.5 cm	13.5 cm	13.5 cm	13.5 cm	15 cm
Length of arm rest	60,5 cm	60,5 cm	60,5 cm	60,5 cm	57 cm
Base width	60 cm	60 cm	60 cm	60 cm	54 cm
Base length	219 cm	219 cm	219 cm	219 cm	188 cm
Total overall width	87 cm	87 cm	87 cm	87 cm	81 cm
Distance between arm rests (adjustable)	60 cm	60 cm	60 cm	60 cm	53 cm
Seat/base height	60 cm	56 cm - 78 cm	60 cm	56 cm - 78 cm	55.5 cm - 57.5 cm
Padding and neck support upholstery	Skai, flame retardant		Skai, flame retardant		
Thickness of padding	8 cm	8 cm	8 cm	8 cm	6 cm
Color frame	light grey	light grey	light grey	light grey	light grey
Color arm and foot rest	dark grey	dark grey	dark grey	dark grey	dark grey
4 castors, antistatic	d = 7,5 cm, all lockable		d = 10, all lockable		d = 7,5 cm, all lockable
Electric motor with limit switch	2 x 24 V	3 x 24 V	3 x 24 V	4 x 24 V	1 x 24 V
Transformer	protection class II		protection class II		
Transformer nominal power	230 V/50 Hz - 110 V/60 Hz		230 V/50 Hz - 110 V/60 Hz		230 V/50 Hz
Primary	230 V/50 Hz - 110 V/60 Hz		230 V/50 Hz - 110 V/60 Hz		230 V/50 Hz
Fuse	1 A/T, slow blow		1 A/T, slow blow		1 A/T, slow blow
Secondary	25 V	25 V	25 V	25 V	25 V
Maximum load	200 kg	200 kg	200 kg	200 kg	150 kg
Net weight	87 kg	88 kg	89 kg	90 kg	48 kg
Gross weight (incl. packaging)	95 kg	96 kg	97 kg	98 kg	55 kg
Dimensions packaging (L x W x H)	800 x 1200 x 1450 mm		800 x 1200 x 1450 mm		600 x 600 x 1800 mm
Back rest tilt	to 62°	to 62°	to 72°	to 72°	to 70°
Seat rest tilt	fixed 10°	fixed 10°	0° to 25°	0° to 25°	
Leg rest tilt	12° to -15°	12° to -15°	2° to -25°	2° to -25°	
Shock position					
Back rest tilt	0°	0°	0°	0°	
Leg and seat rest tilt	to 14°	to 14°	to 27°	to 27°	
Room temperature	3°C - 40°C		3°C - 40°C		3°C - 40°C
Relative humidity	30% - 75%		30% - 75%		30% - 75%
CE/MDD	CE/Class I		CE/Class I		CE/Class I

Ordering information

For more information such as literature, technical details and working procedures, as well as for equipment, please contact your local sales representative or us.

Order number Compocouch models

9020521	Start A2 with wedge shaped armrest, lightgrey
9020531	Start A2 with wedge shaped armrest, pastel blue
9020561	Start A3 with wedge shaped armrest, lightgrey
9020571	Start A3 with wedge shaped armrest, pastel blue
9020601	Evolution A3 with wedge shaped armrest, lightgrey
9020611	Evolution A3 with wedge shaped armrest, pastel blue
9020641	Evolution A4 with wedge shaped armrest, lightgrey
9020651	Evolution A4 with wedge shaped armrest, pastel blue
9024751	Compochair with wedge shaped arm rest
9024501	Compochair standard, grey

Order number Accessories

5067371	Infusion pole
9020701	Infusion pole holder
9020711	Start / Evolution - battery option
9020721	Start / Evolution - LED reading lamp, 24 V
9020731	Start / Evolution push bar, fixed
9020741	Start / Evolution relax pillow, pastel blue
9020751	Start / Evolution relax pillow, lightgrey
9028141	Side table ST1 with motionless tray
9028131	Side table DT1 with tray
9020761	Transparent plastic cover for foot part
9020771	Foot rest - mechanically adjustable
9020781	Start / Evolution - foot rest option

All couches equipped with Paper roll holder

"Evolution" models with relax pillow

"Start" models: simplified base construction, armrest without height adjustment and additional neck role



**FRESENIUS
KABI**

caring for life

Fresenius Kabi Deutschland GmbH
Else-Kröner-Straße 1 · 61352 Bad Homburg · Germany
T +49 6172 686-5047 · F +49 6172 686-4728
www.fresenius-kabi.com

Your local sales organization: