



**FRESENIUS
KABI**

caring for life

How to connect the Freka[®] Belly Button extension set


Instructions for use



1 Remove the Freka Belly Button extension set from its packaging.




2 Open the sealing tab of the Freka Belly Button. Line up the black lines on the Freka Belly Button and the Freka extension set.

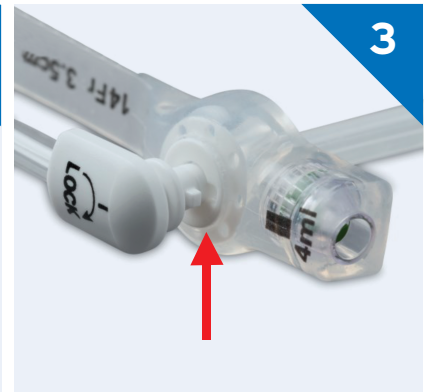


3 Ensure the extension set is correctly lined up with the feeding port on the Freka Belly Button, before pushing the end of the extension set into the feeding port.



4 Gently rotate the extension set 90° clockwise, as indicated on the connector.

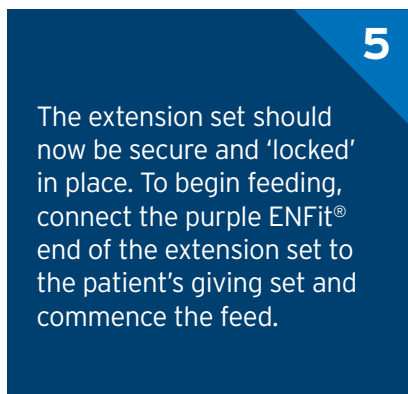

5 The extension set should now be secure and 'locked' in place. To begin feeding, connect the purple ENFit[®] end of the extension set to the patient's giving set and commence the feed.

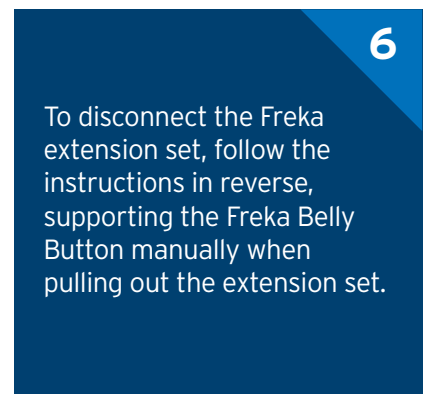

6 To disconnect the Freka extension set, follow the instructions in reverse, supporting the Freka Belly Button manually when pulling out the extension set.