


## What to do if...

### If alarm occurs:

 Mute alarm (if necessary)


Detect the cause and proceed to the appropriate action

 Release alarm  Resume feeding

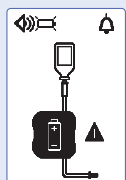


#### Empty battery

Connect pump to mains supply immediately.

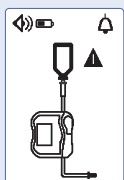


#### Empty bag/ air in set alarm

Change feed container, check connection to giving set and ensure giving set is fully primed.

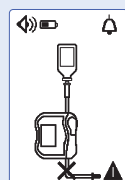


#### Downstream occlusion alarm

Check that pumpset/feeding tube is clear of any occlusion after the pump, at the patient side.

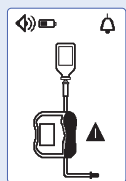

#### Door open alarm

Close pump door.

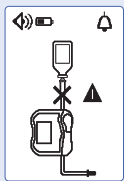


#### Upstream occlusion alarm

Check that pump set is clear of any occlusions between the container and the pump




#### Target volume almost reached message

Continue feeding or end feeding.

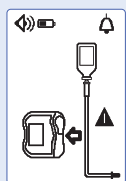


#### Set installation alarm

Check giving set position above and below pump mechanism and insert it correctly

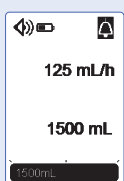


#### Target volume reached alarm

End feeding or proceed to the next step

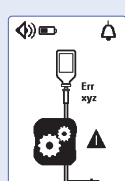

#### Pump error alarm

Note the technical Error code (Err xyz) and contact your biomedical department.

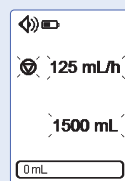

#### Start Reminder

Proceed to next step or switch pump off.



**FRESENIUS  
KABI**

caring for life

# Amika®

## ENTERAL NUTRITION PUMP PICTORIAL REFERENCE GUIDE



FOR ANY ENQUIRIES OR INFORMATION CALL THE HOMECARE  
24 HOUR FREEPHONE HELPLINE NUMBER 0808 100 1990



**FRESENIUS  
KABI**

caring for life

Fresenius Kabi Limited,  
Cestrian Court, Eastgate Way,  
Manor Park, Runcorn,  
Cheshire WA7 1NT  
tel: 0808 100 1990

[www.fresenius-kabi.com/gb](http://www.fresenius-kabi.com/gb)

Date of preparation: October 2023 GB-Ami-2300001

Amika® is a registered trademark of Fresenius Kabi AG.  
Fresenius Kabi Ltd is an authorised user © Fresenius Kabi Ltd.

SETTING UP THE PUMP

01



Close the pinch clamp on the giving set and then connect to the EasyBag or feed container.

02



Open the pump door.

03



Place the giving set pinch clamp in line with the downward arrow until it "clicks" in place.

04



Place the giving set tube straight inside the tube guide.

05



Close the door, push down the lever to lock it firmly in place.

06



Turn the Amika® pump on - pressing the on/off button.

07



Press the automatic priming key, and then press the start button.

08



When priming is complete, press the start button again.

09



Set the desired flow rate by pressing the +/- keys until the desired volume is reached.

10



Set the target volume by pressing the +/- keys until the desired volume is reached.

11



Connect the giving set to the patient's enteral feeding tube. Press the start key to start feeding.

12



To stop feeding at any point, press the stop key. When feeding has finished or your target volume is reached, press the stop key.

13



If you need to reset the progress bar after target volume is reached, press the stop key for 2-3 seconds.

14



To switch off the pump press the on/off button until the countdown is completed.

